

## **CANON Upgrade procedure, Canon NE 3.11 to 3.12**

This instruction describes the procedure to upgrade from Canon NE 3.11 to Canon NE 3.12.

- 1) Run Product installer 3.12.1.2
- 2) Run Patch 3.12.1.3
- 3) Run protocol export tool
- 4) Run Migration collection tool to save the current settings of the latest 3.12.1 version
- 5) Uninstall the current (3.12.1) version completely.
- 6) Reboot PC !
- 7) Run Product installer 3.12.1.2
- 8) Run Patch 3.12.1.3
- 9) Install options, like BiAA and INR
- 10) Put back the saved 3.12.1 migration data, now you have a clean install with the initial settings done in 3.10 or 3.11