



1. Release and remove the screws (4pcs)...

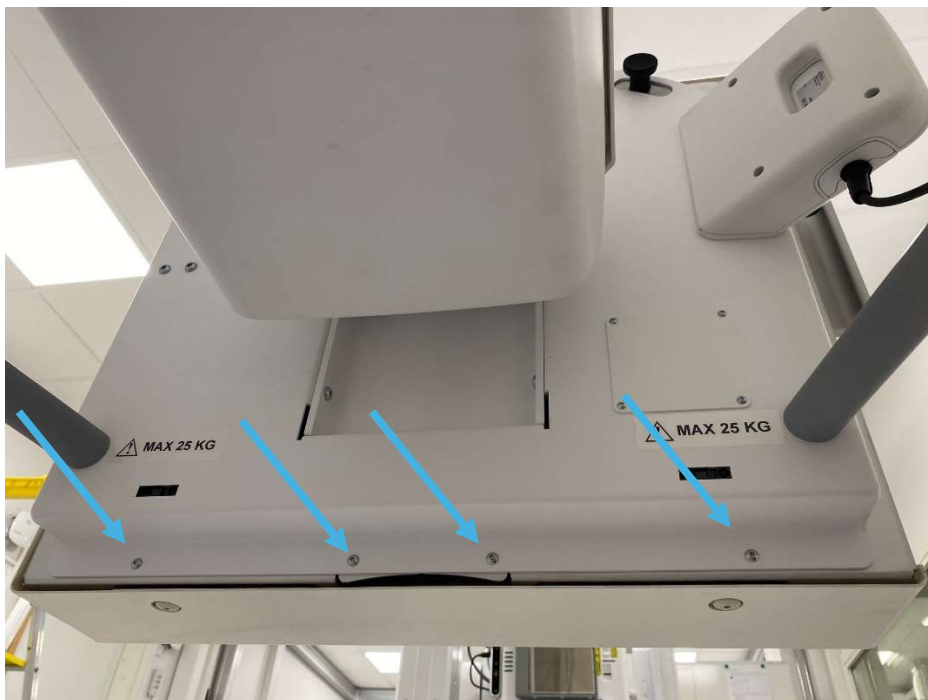

2. Rest the bucky on a suitable, stable object

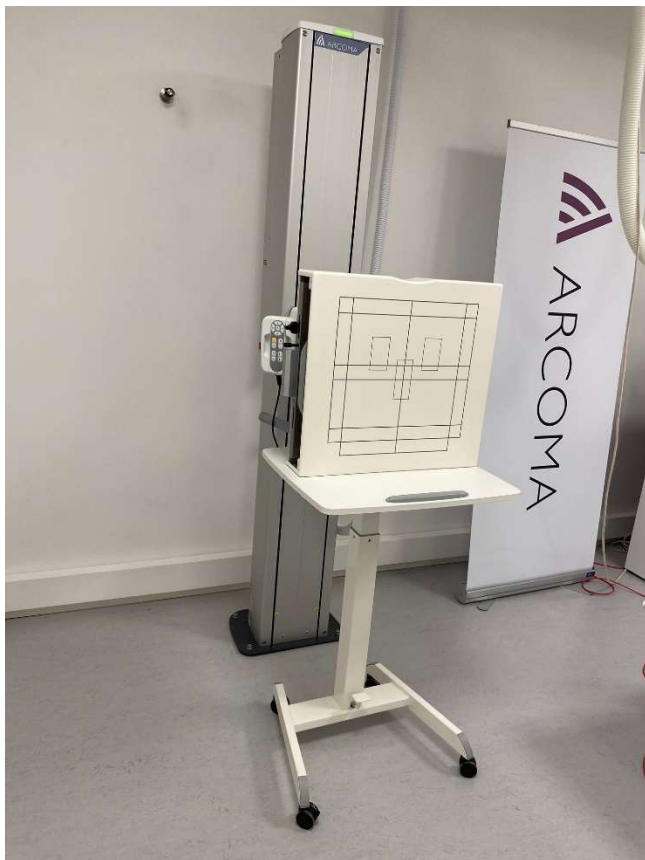

**NOTE!** Be careful during the next steps. The wall stand will be highly unbalanced when relieved of the detector holder weight.

3. Release and remove screws to separate the bucky from the stand.



4. Carefully separate the bucky from the stand. Cut any cable ties and disconnect the cable for the microswitch.



5. Remove the small cover.
6. Release and remove the grub screw.



7. Using a mounting tool unscrew and remove the tap handle



8. Release and remove the screws.



9. Pull the block with the microswitch out.



10. Replace the microswitch.

11. Remount in reverse order.