

UDI_0073-815-007 Battery Holder, Remote control

How to install the Battery holder kit in Advanced Remote control.

APPLICABLE TO:


Precision i5



SPARE PART: 0073-815-007

TOOLS NEEDED:

- Standard Service tools.
- Torx tool size T6.

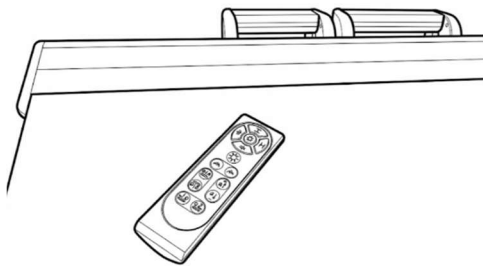

PROTECTIVE MEASURES:

CAUTION!

Printed circuit boards contain electrostatic highly sensitive components requiring particular care in their handling. Ground before making contact and place only on a conductive surface.

CAUTION!


Remaining energy may exist when the equipment is switched off. Always wait at least 15 seconds before working on the system

ACTION STEPS:

1. Remove the plugs from the four screw holes on the back of the remote control.



2. Release and remove the four screws. Remove the back cover.

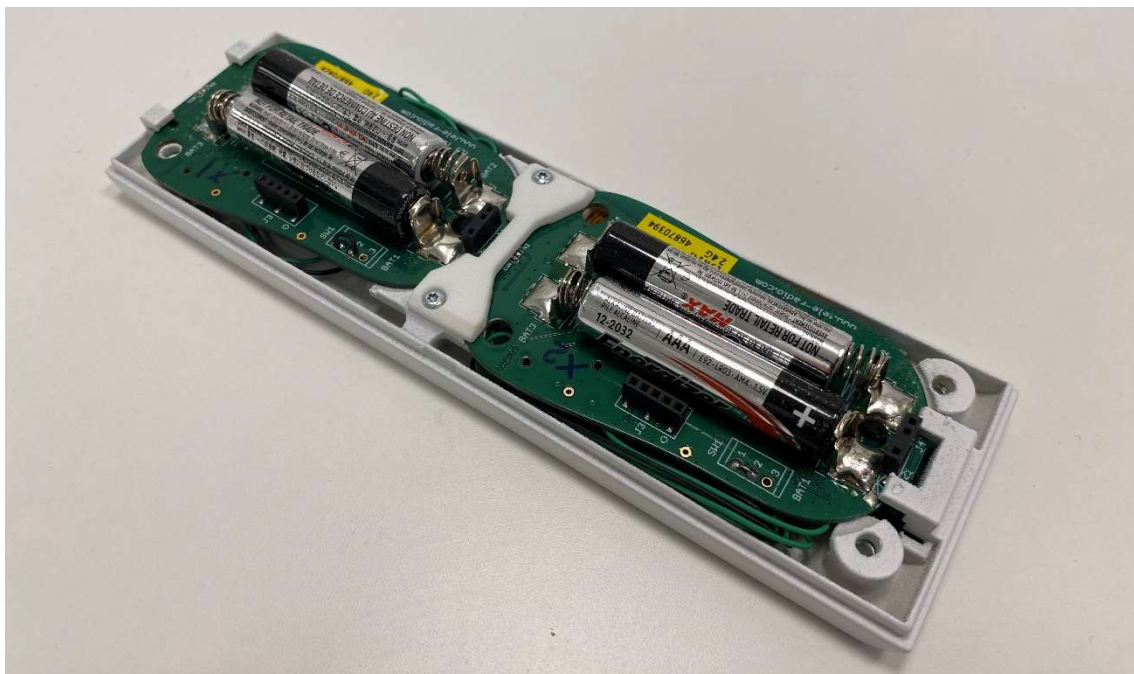


3. Remove batteries and position the battery holder. Insert batteries.



4. Repeat for the second PCB.



5. Release the two screws for the original PCB holder and remove it. Install the new PCB holder from the upgrade kit. Existing screws can be used but some are delivered with the upgrade kit if needed.

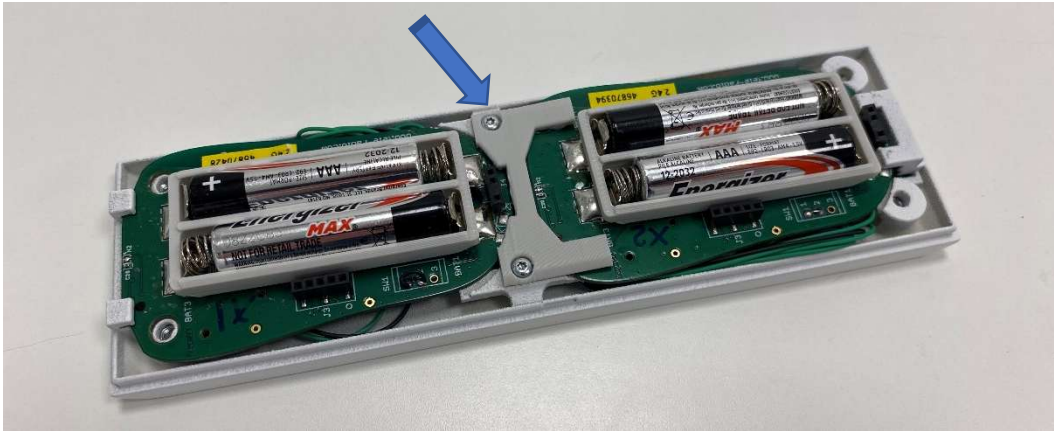

6. Reinstall back cover and plugs.
7. Test the remote control to confirm that it works as expected after the upgrade. No re-pairing transmitter and receiver is required.

END OF DOCUMENT