



Spare Part Catalogue



Arcoma Intuition / Arcoma Intuition i3

AIM

The Spare Part Catalogue is designed to help identify the required spare part and its corresponding unique spare part/article number. This number is crucial when placing an order for a replacement part to Arcoma.

For pricing information, please refer to the Spare Part Price List (SPL).

HOW TO NAVIGATE THIS SPARE PART CATALOGUE

The spare parts are organized based on their location within the system. You can locate the correct part either by using the search function in the PDF or by navigating to a specific section through the links below:

- [Overhead tube crane \(OTC\)](#)
- [Wall stand \(WS\) + Wall stand bucky](#)
- [Table \(TS\): Two column table + Table bucky](#)
- [Table \(TS\): Closed table](#)
- [System cabinet](#)
- [Imaging Acquisition System and Parts in Control Room](#)

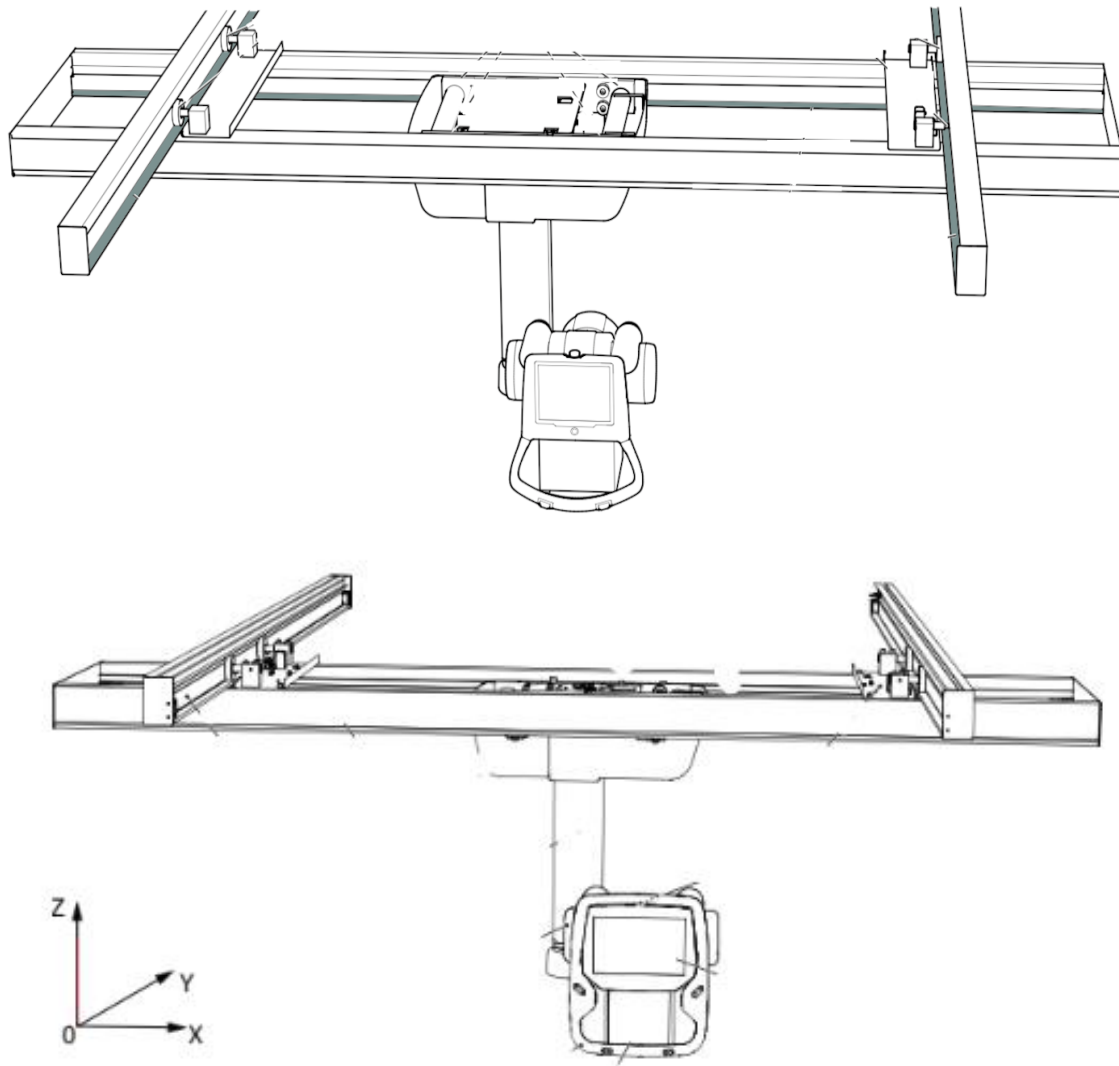
Each spare part has a description, spare part/article number, drawing, and photo. If the spare part includes additional visible information or numbers, these are also listed to simplify identification. See example below. The spare part / article number shall be used when placing an order for a replacement part.

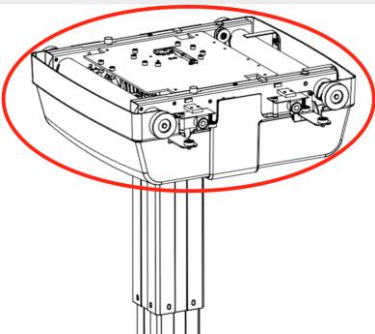
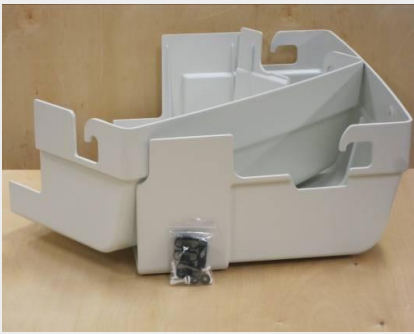
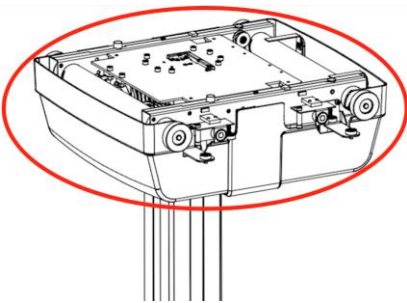
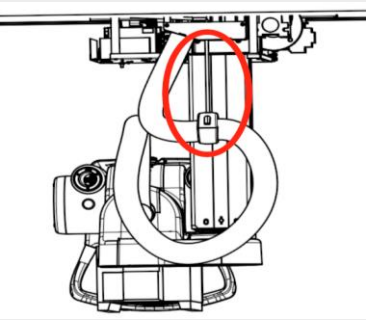
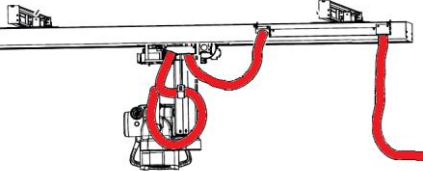
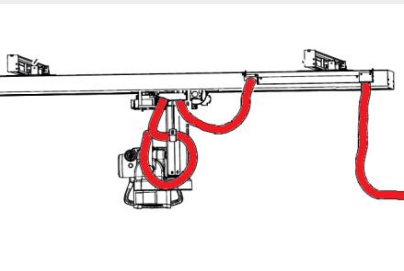
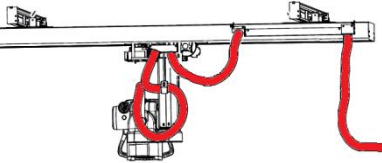
Description	Spare Part Article No.	Info/Number displayed on part	Drawing	Photo
Brake Magnet A	0070-810-019	1.AMAG01 0070-004-019		

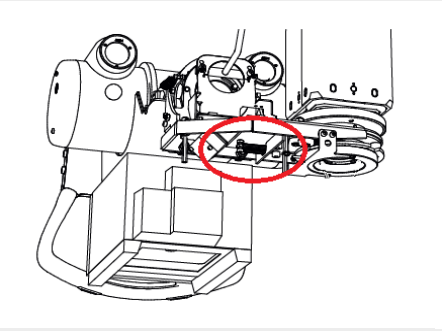
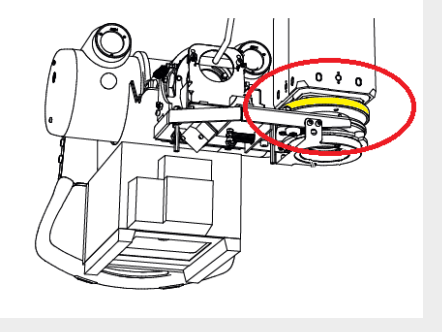
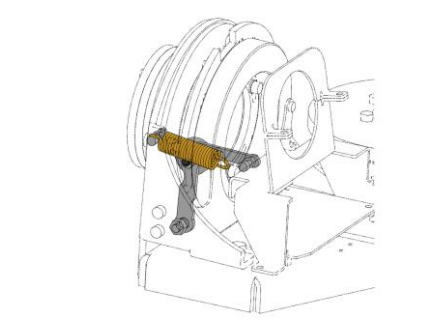
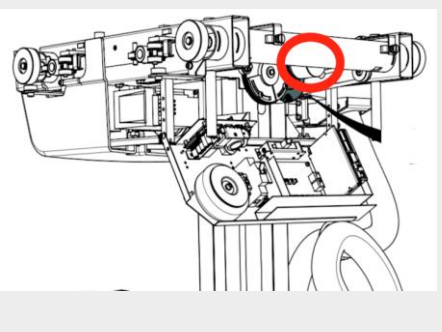
If an instruction is available on the Arcoma Partner Portal, it is marked with a blue icon, as shown below.

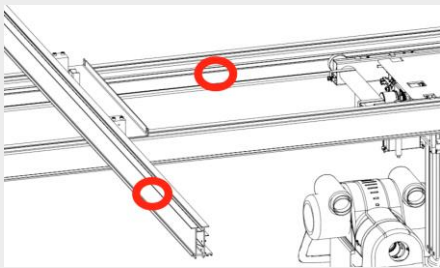
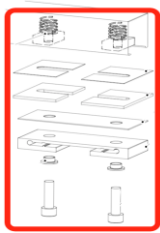
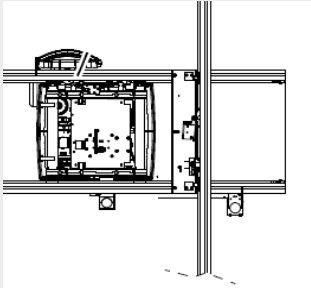
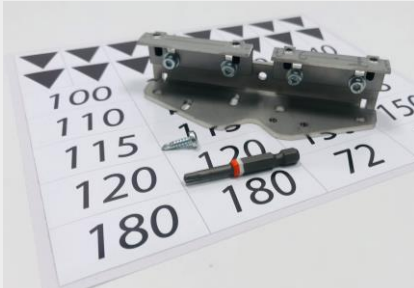
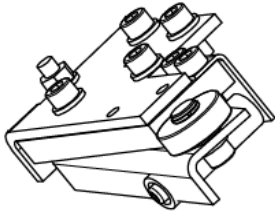
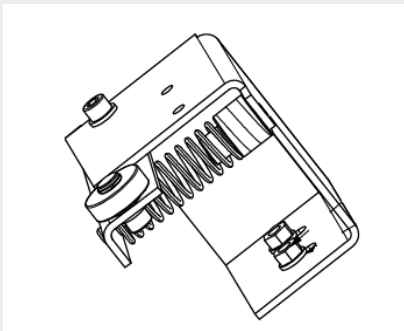


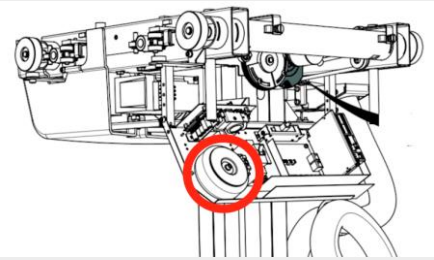
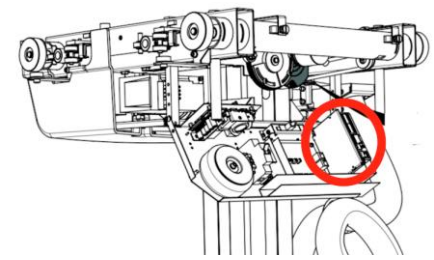
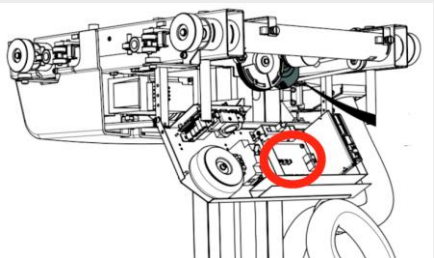
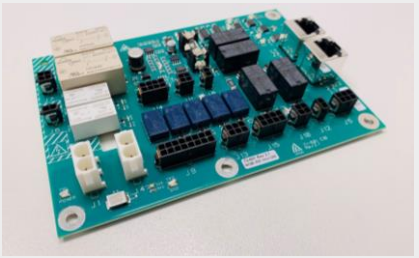
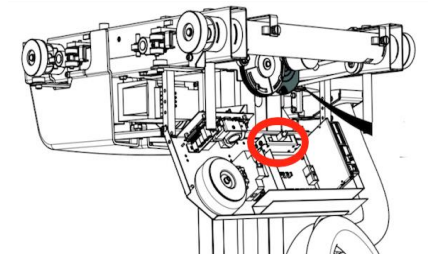
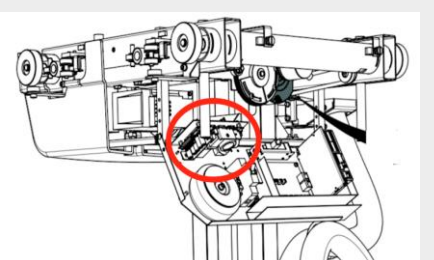
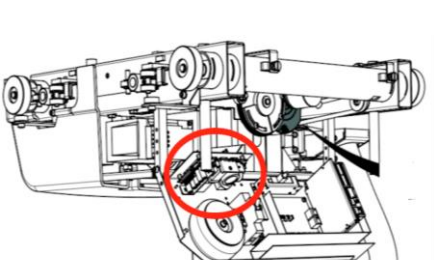
OTC - Overhead Tube Crane

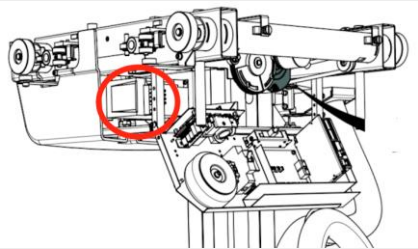
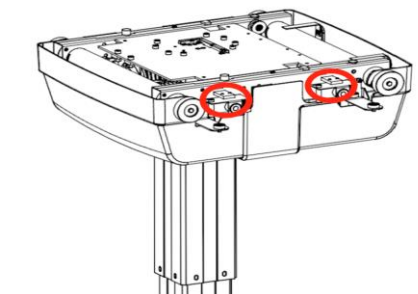
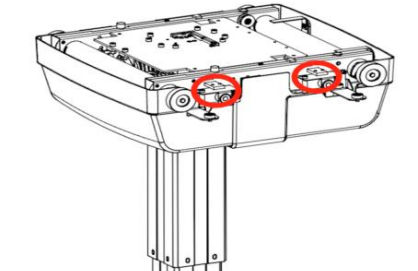
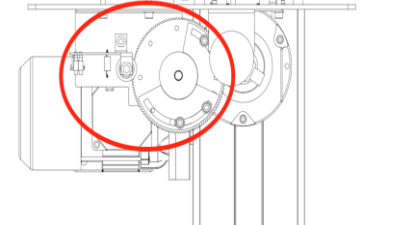
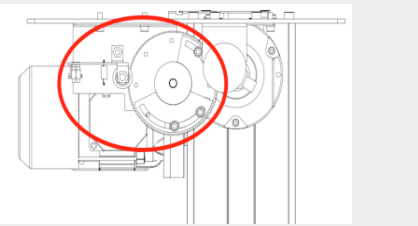
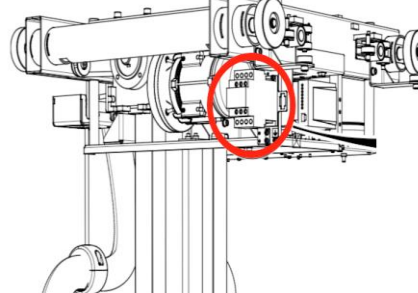


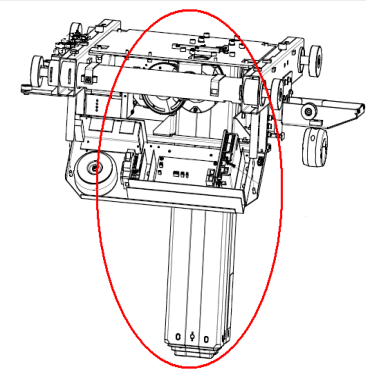
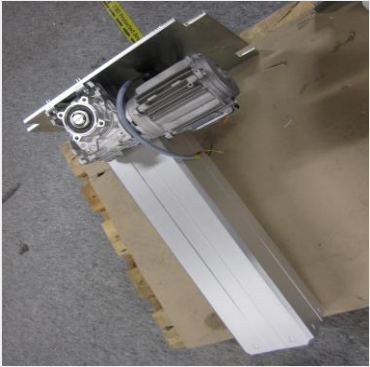
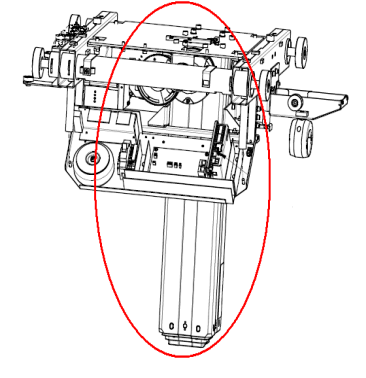
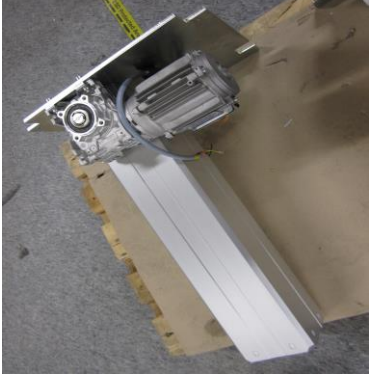
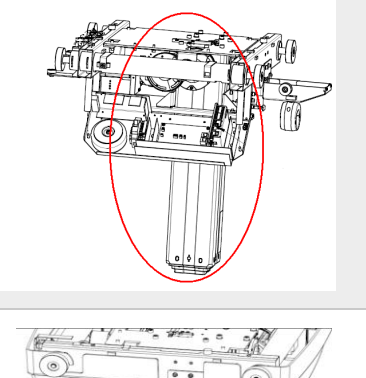
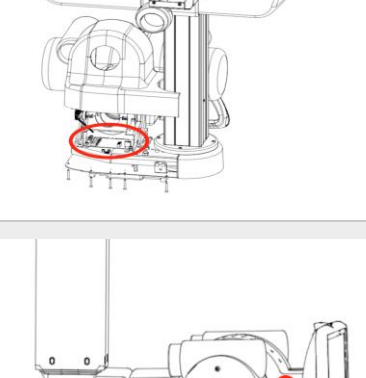
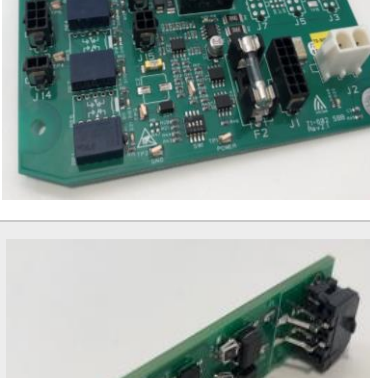
Description	Spare Part article no	Info/Number on spare part	Drawing	Photo
Covers ceiling wagon OTC	0170-810-223	0170-605-023 0170-605-024 54-071 <i>FastLoc kit included</i>		
FastLoc kit For Ceiling wagon covers	0200-810-071	54-071		
Elastic band	0170-810-009	-- <i>Elastic band, plastic clamp + plastic screws and washers</i>		
HV Cable assembly 16m	0183-810-316	0183-750-316		
HV Cable assembly 24m	0183-810-324	0183-750-324		
Protective hose, For OTC HV cable assy.	05-052 Length defined at the time of order.	<u>Intuition (16m):</u> Tube - OTC 2500mm OTC - X 2000mm X - Wall 4200mm <u>Intuition (24m):</u> Tube - OTC 2500mm OTC - X 2000mm X - Wall 4700mm		

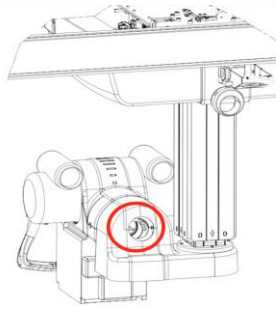
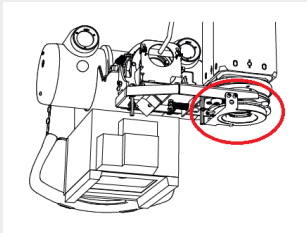
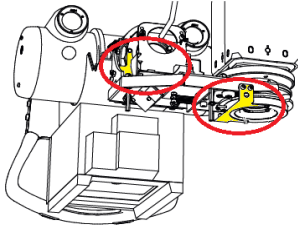
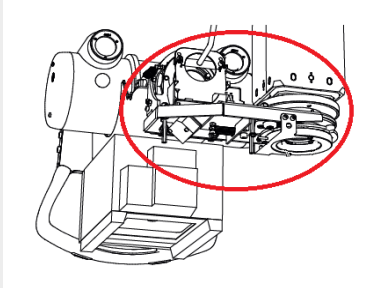
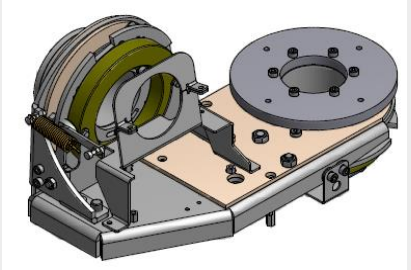
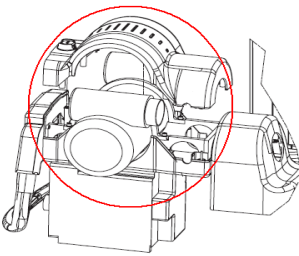
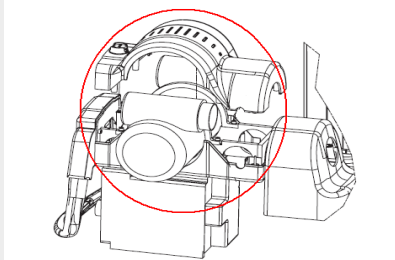
Index assy. cpl. Beta	0170-810-014	0170-004-014 904615-0x		
Index plate Alpha/Beta	0170-810-008	0170-004-008		
Label, angle Beta	0170-810-399	0170-003-399		
Index assy. Cpl. Alpha	0170-810-923	0070-004-015 904614-0x (923074)		
Encoder Z	0170-810-040	0500-154-040 1.ZABS01		

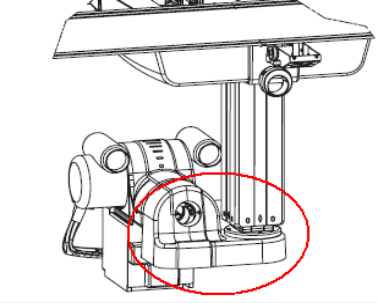
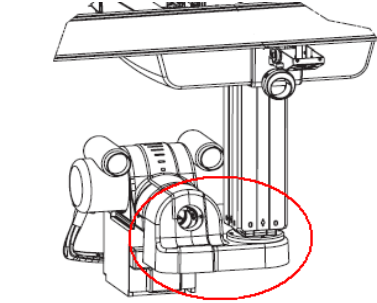
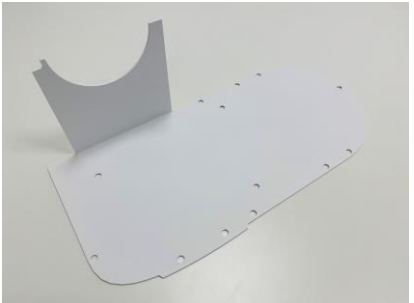
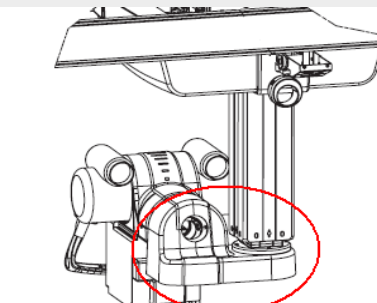
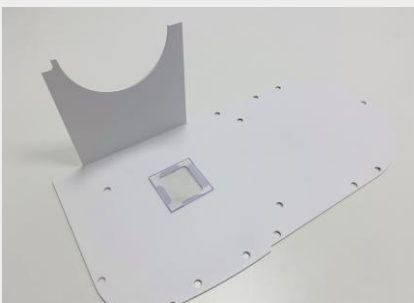
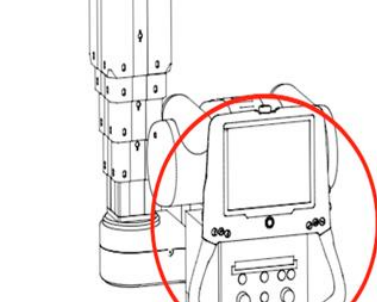
Endstop adjustable, traverse	0170-810-093	<i>Adjustable end stops to be located anywhere on X- and Y-rails (to limit movement)</i>		
Spare part Y-rail attachment kit	0170-810-020	0170-003-005 0170-100-010 0170-100-011 54-040 MC6S M10x35 A BRB 10,5x21x2		
Mech Index	0170-810-370	0170-151-370 68-012 54-403		
Mech Index Ceiling wagon (X)	0170-810-371	0170-151-250 MC6S M6x14 A BRB 6,4x12,5x1,6		
Mech Index Traverse (Y)	0170-810-372	0170-151-360 BRB 6,4x12,5x1,6 M6M M6		
Cross traverse carriage	0170-810-033	0070-001-235 0170-002-037 0170-003-012 SGA 20 SGA 12 SK6SS M4x8 D		

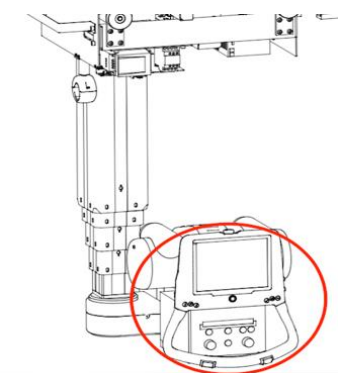
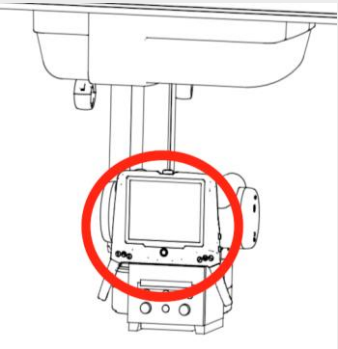
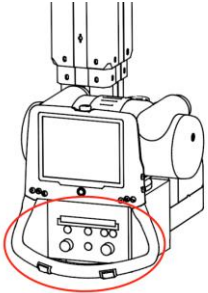
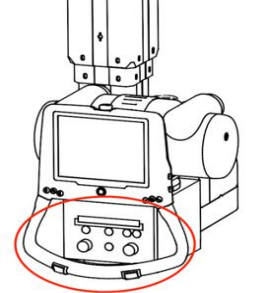
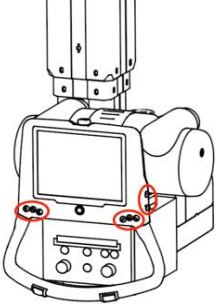
Transformer, Collimator	0170-810-066	12-066 Only used for OTC's with halogen type collimator light		
PCB CB Master/Z node	0170-810-101	70-106 1.1CB01		
PCB CIB	0170-810-601	72-601 1.1CIB01		
BCM connection box	0500-810-001	0048-099-001 1.1BCM01		
Power supply 24V	0512-810-408	12-408 1.1PS01 For s/n 2001-80100.		
Power supply 24V	0073-810-033	12-033 1.1PS01 For 80101 and above.		

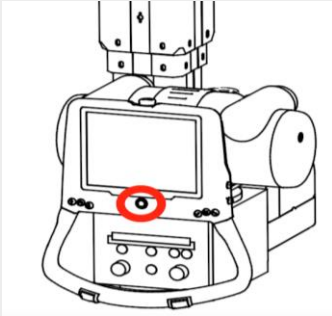
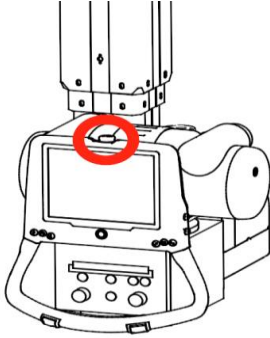
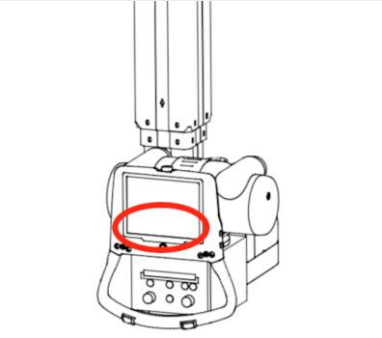
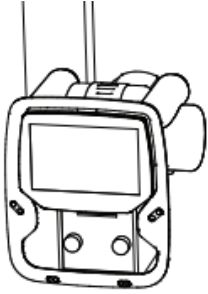
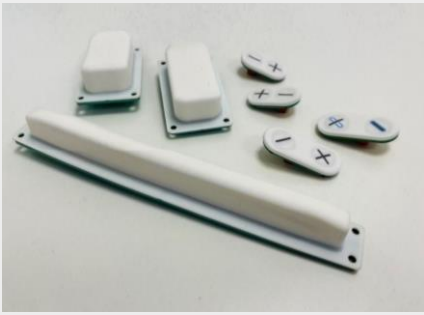
Frequency converter ATV12	0180-810-002	13-062 1.2FRE01		
Brake unit X/Y	0170-810-238	0170-151-239 1.3BRA01 1.3BRA02 1.4BRA01 1.4BRA02		
Guide wheels, kit	0170-810-074	0170-002-074 SGA 8		
Height sensor "Safety height"	0170-810-240	0170-151-240 For: 1700 mm column.		
Height sensor "Safety height"	0170-810-241	0170-151-241 For: 1450 mm column.		
Relay 1.2RE01 (20A)	0170-810-056	10-056 1.2RE01		

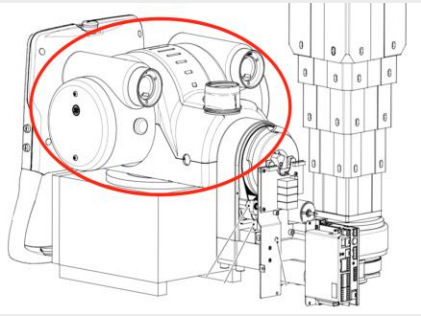
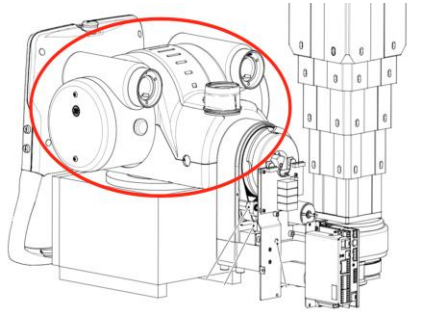
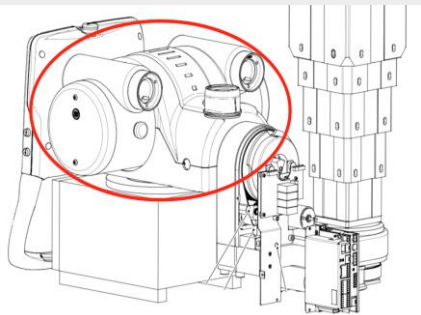
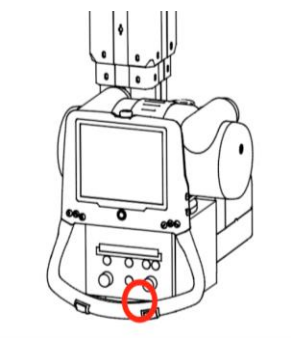
Column Z 1450 mm stroke	0170-810-305	50-305		
Column Z 1700 mm stroke	0170-810-306	50-306		
Cleaning and lubrication kit for Z column	0500-810-875	-- Intended for lubrication of Z column		
PCB SBB	0180-810-300	72-300 1.5SBB01		
PCB OIB	0184-810-227	72-227 1.3OIB01 <i>Located underneath tube cover. Only used with 10" display.</i>		

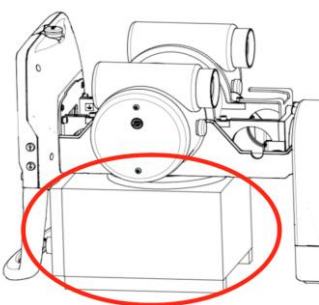
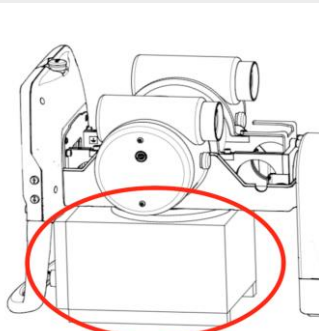
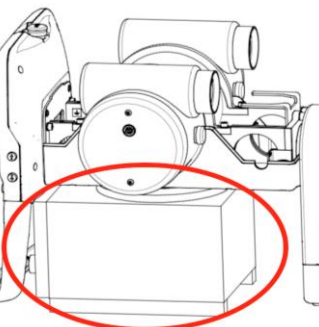
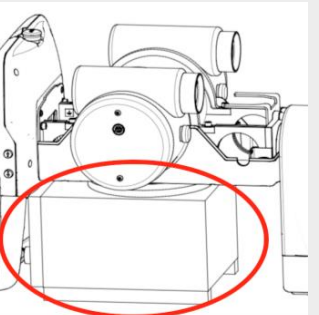
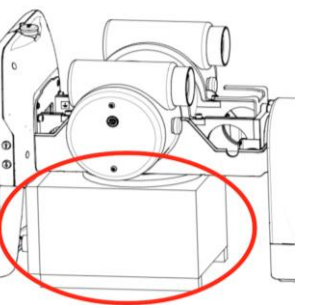
Microswitches Alpha	0170-810-002	0170-750-401 02-055		
Magnet Alpha/beta	0170-810-006	17-007 1.ABRA01 1.BBRA01		
Bracket, magnet	0170-810-022	25-008 K6S M4x6 A KB 4,1x8,2		
Rotation assy. A/B cpl.	0170-810-400	0170-151-400		
Tube covers, RAD14 tube	0170-810-225	0170-605-025 0170-605-030		
Tube covers, A292/G292 tube	0170-810-235	0170-605-022 0170-605-027 Replaced by 0183-810-235 for 4C and i3.		

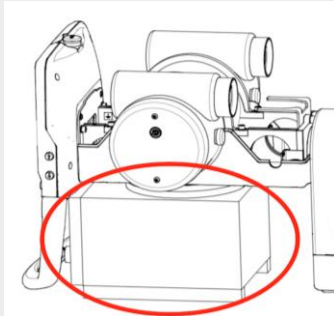
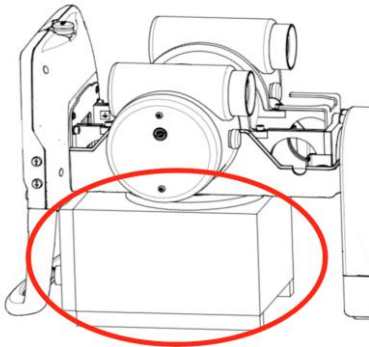
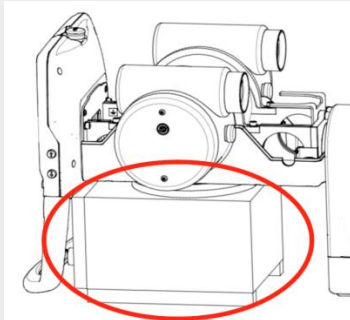
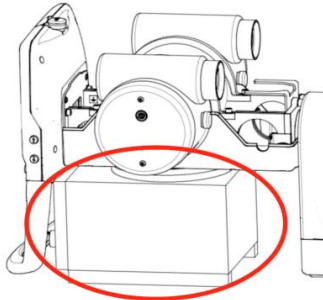
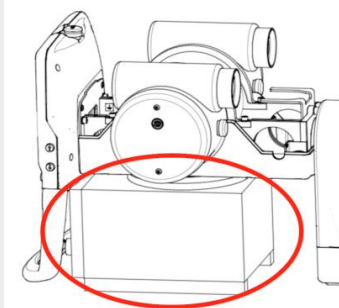
Tube covers, A292/G292 tube	0183-810-235	0170-605-022 0183-605-003 0073-605-017 4C and i3		
Cover A/B	0170-810-232	0170-605-032 0170-605-035 0170-605-103		
Cover, lower Alpha/Beta	0170-810-505	0170-605-005		
Cover, lower Alpha/Beta IR	0170-810-046	0170-605-046		
Maneuver Display complete, with loop Light blue – color code: 625	0183-810-009	0183-151-004 Arcoma blue Code 625 3 rd generation		

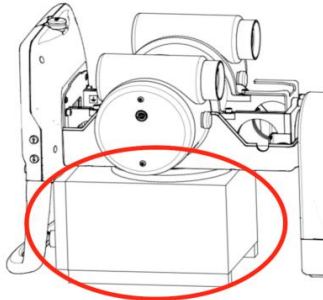
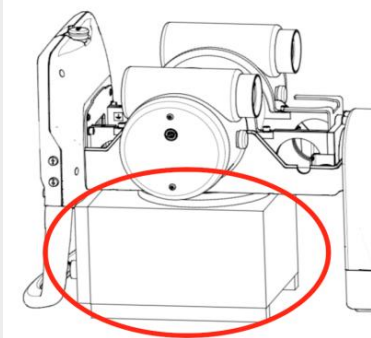
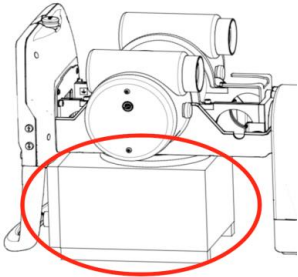
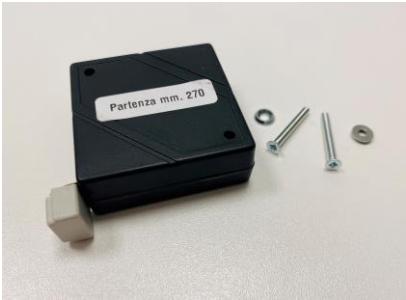
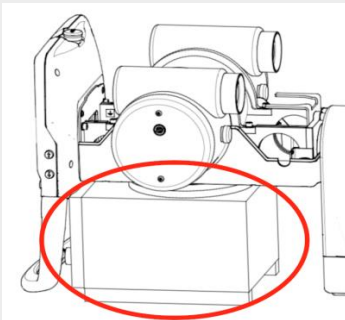
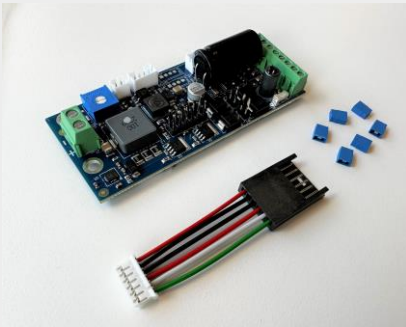
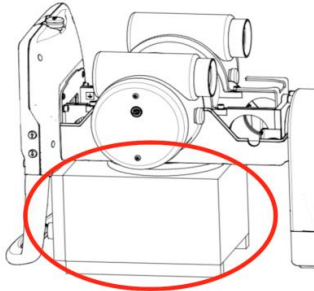
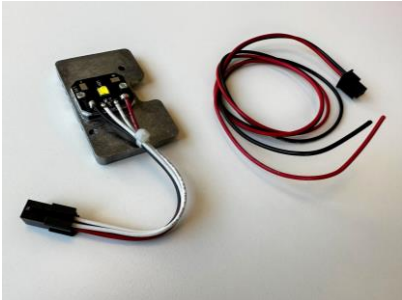
Maneuver Display complete, with loop Grey – color code: 629	0183-810-011	0183-151-006 Option color code -629 3 rd generation		
Maneuver Display complete, with handles Light blue – color code: 625	0183-810-012	0183-151-005 Option color code -625 Blue 3 rd generation		
Handle frame X/Y/Z Light blue – colour code: 625 3 rd generation	0073-810-030	0073-154-027 02-632 M6 Nylock x2		
Handle frame X/Y/Z Grey – color code: 629 3 rd generation	0073-810-031	0073-154-028 02-632 M6 Nylock x2		
Buttons, maneuver handle / display Seal kit	0070-810-016	0070-007-261 0070-007-262 0070-007-263 0182-003-260 0182-003-261 0182-003-263		

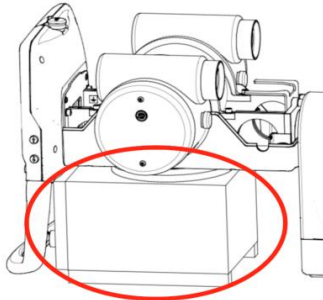
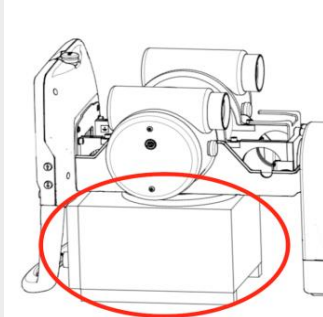
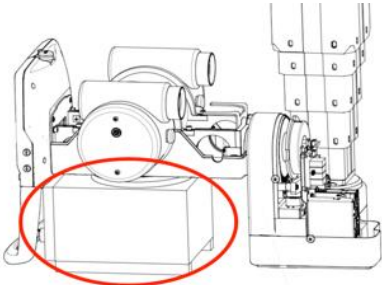
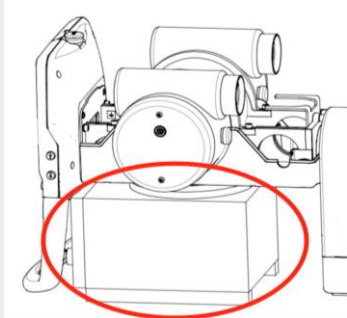
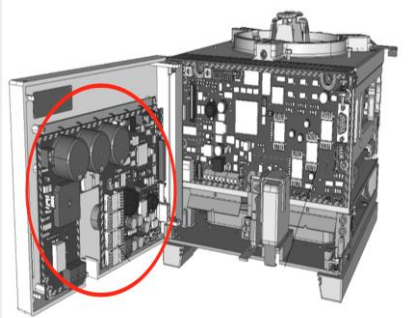
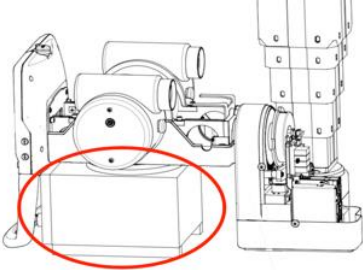
Cable, Servo button	0073-810-019	06-500 1.3J04		
Cable, Emergency Stop	0073-810-009	0073-750-202 1.3J01 1.3EM01 Used in 4C and i3.		
PCB SDB, CAN bus module inside display assembly 4C	0180-810-014	72-166 1.6SDB01		
Manoeuvre Display 12" 0180 i3	0180-810-220	0073-154-221 Used from Intuition s/n 80100. Incl. accelerometer.		
Button set, Display 0180 i3	0073-810-066	0073-154-031 1.3FBL01 1.3FBR01 1.3FBL02 1.3FBR02 1.3BBL01 1.3BBB01 1.3BBR01		

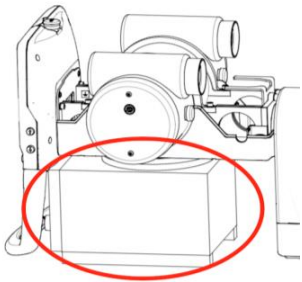
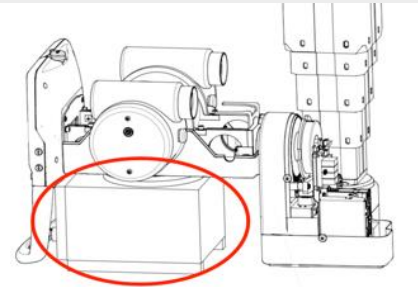
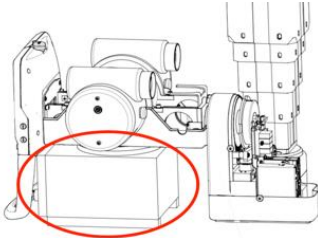
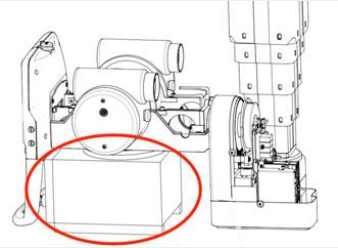
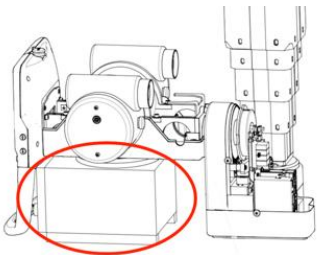
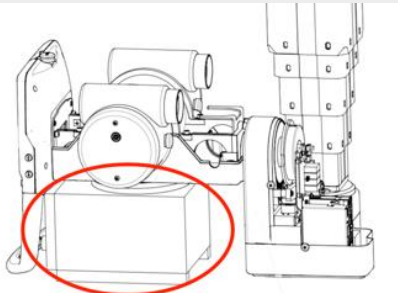
Cable DSP signals	0183-810-010	0183-750-010		
X-ray Tube A292 400 kHU RAL 9003	1105-810-110	90-113		
X-ray Tube G292 600 kHU RAL 9003	1105-810-114	90-114		
X-ray Tube RAD14 300 kHU RAL 9003	1105-810-113	90-116		
Tape measure AL02 II and eL Collimators	0200-810-780	90-780		

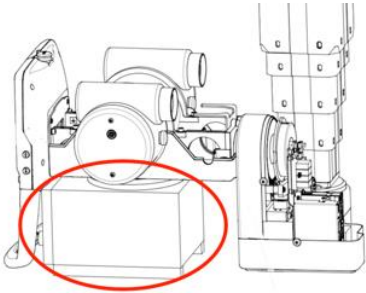
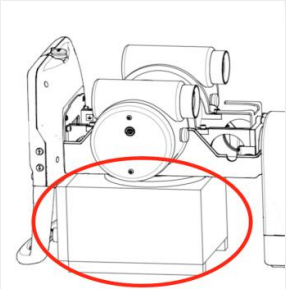
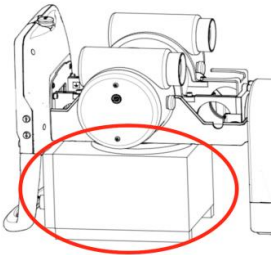
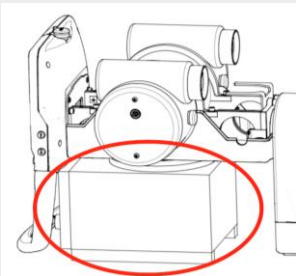
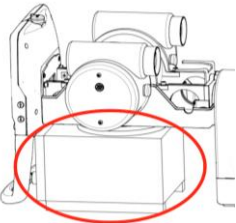
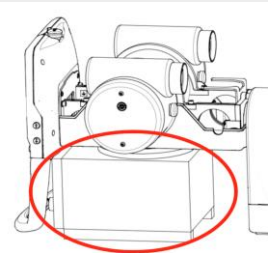
Collimator AL02-2 Halogen	1105-810-200	90-200 Discontinued. Has been replaced by 1105-810-206.		
Collimator AL02-2 LED	1105-810-206	90-205		
Collimator, R221 ACS	1105-810-229	90-229 Discontinued. Has been replaced by 1105-810-231.		
Collimator, R221 ACS DHHS	1105-810-231	90-230 Can be used to replace 90-229. New rotating tube flange p/n RS 2259 required.		
Front Panel R230 ACS	RS 2049	--		

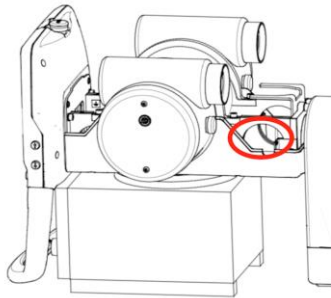
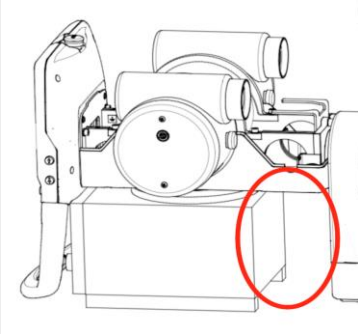
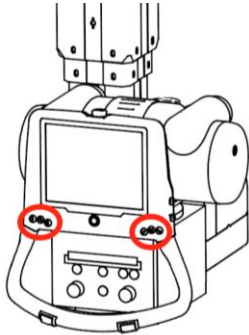
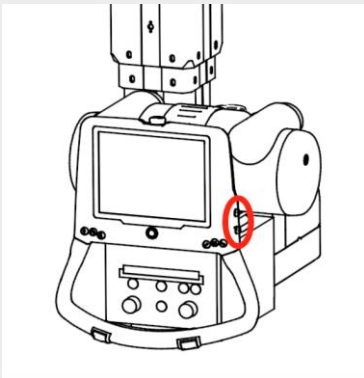
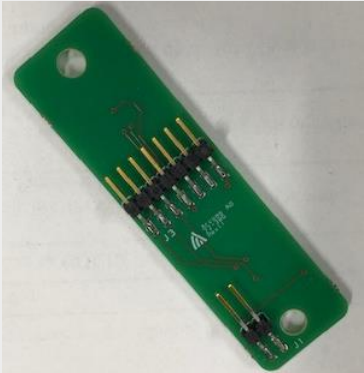
Front Panel R221 ACS	RS 2698	--		
Collimator, Ralco 302/A DHHS Cust.: 051D	1105-810-226	90-226		
Front Panel Ralco 302/A Cust.: 051B/D	RS 1555	RS 1372 frame included		
Timer Board GC 338 Ralco 302/A Cust.: 051B	RS 063	Can be used to replace the discontinued version FM431. See instruction for re-wiring.		
Timer Board FM431 Ralco 302/A	RS 308	Discontinued but available as spare part. Can be replaced by GC 338.		

Light bulb R302 Collimator	RS 006	24V, 100W		
Laser, Collimator R302 Collimator	RS 409	--		
Tape measure R221 ACS / ACS DHHS	0200-810-878	RS 1878		
LED Board R221 ACS		RS 2539/B Timer board GC_LED-4A		
LED unit R221 ACS / ACS DHHS	RS 2748	Power LED Can be used to replace RS 1065 which is no longer available.		

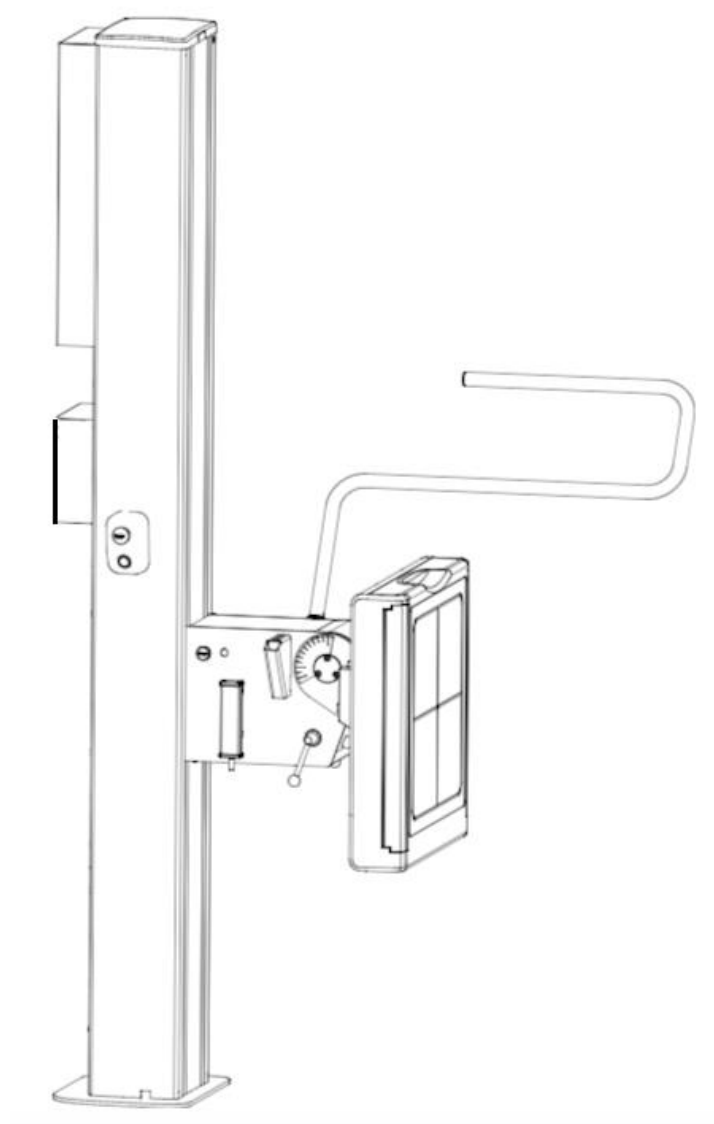
Laser R221 ACS / ACS DHHS	RS 2589	--		
Fan AL02 Collimator	1105-810-070	25-070 10092689		
D22, Front panel board AL02 II Collimator		08628120		
PCB D22eL for AL02 LED Collimator	1105-810-074	71-218 10591909 For AL02 s/n <8000, Rev 02 or lower.		
PCB D23eL AL02 II eL LED Collimator	1105-810-075	71-219 11021083 For AL02 s/n >8000, Rev 03 or higher.		

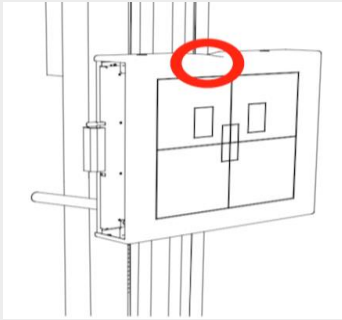
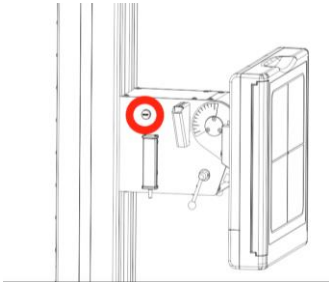
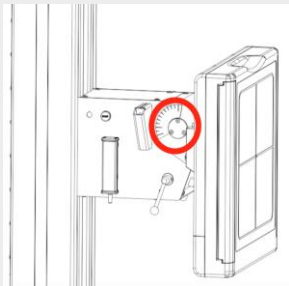
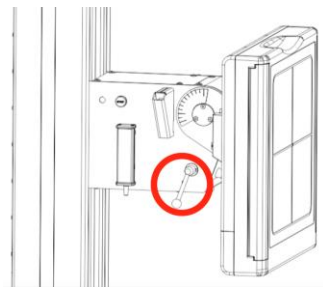
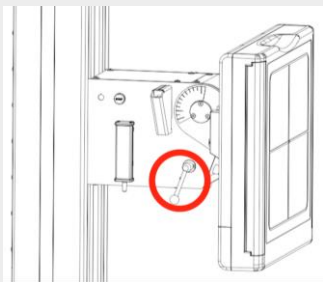
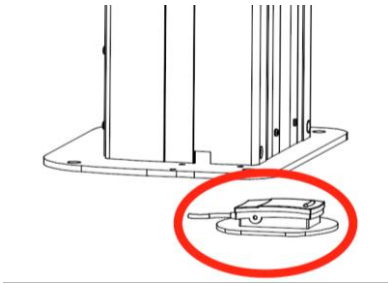
Thermal Switch AL02 Collimator	1105-810-076	20-076		
Lamella, Focus near assy. AL02 II and eL Collimators	1105-810-077	25-152 07736759		
Heat shield AL02 II Collimator	1105-810-078	25-078 10092665		
Cover panel, left AL02 II and eL Collimators	1105-810-150	25-150 10642867		
Cover panel, right AL02 II and eL Collimators	1105-810-151	25-151 10642866		
Cover, Halogen lamp AL02 II Collimator	1105-810-205	19-027 10092545		

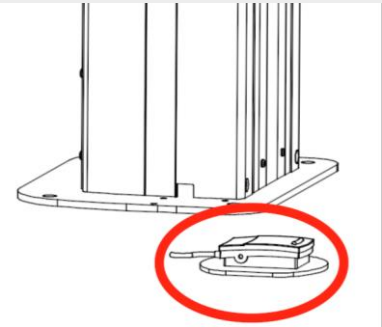
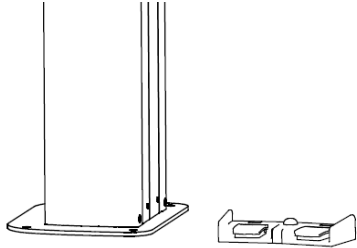
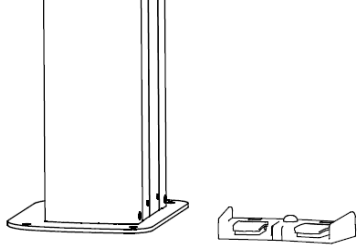
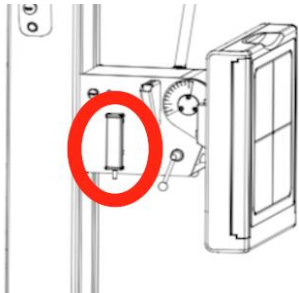
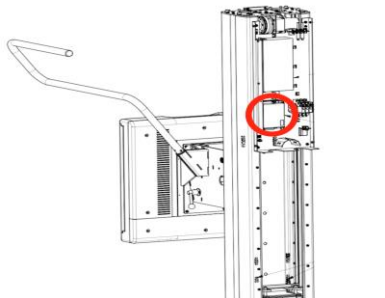
Cover, LED AL02 II eL Collimator	1105-810-207	10398379 19-031		
DAP Chamber Standard Sensitivity	1105-810-301	90-301		
DAP Chamber High Sensitivity (HS)	1105-810-306	90-306		
Light bulb AL02 Collimator	1105-810-545	19-026		
Laser AL02 Collimator	1105-810-892	16-002		
LED module AL02 Collimator	1105-810-890	16-002		

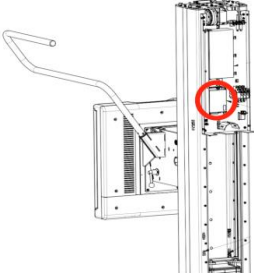
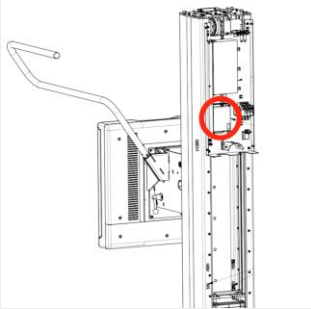
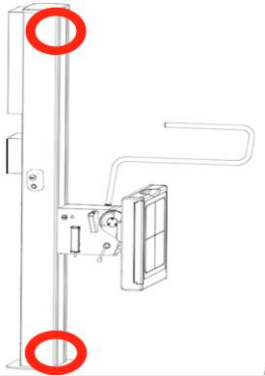
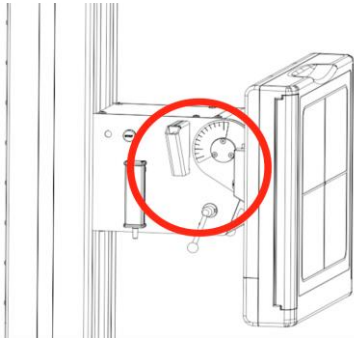
Adapter Cable AL02 Collimator	0073-810-024	0073-750-004		
Main Cable AL02 Collimator	0073-810-023	0073-750-003		
Handle buttons PCB 3 rd generation	0070-810-018	72-305		
Handle buttons PCB, Up/down 3 rd generation	0070-810-022	72-306		

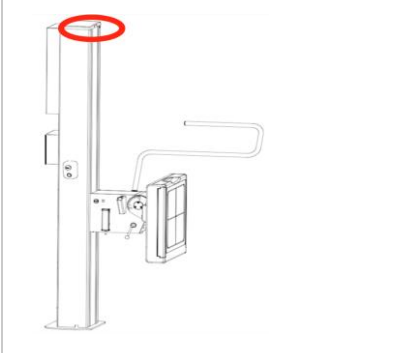
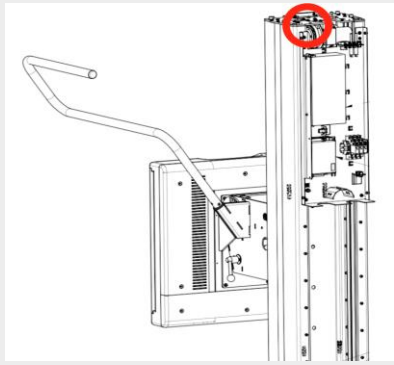
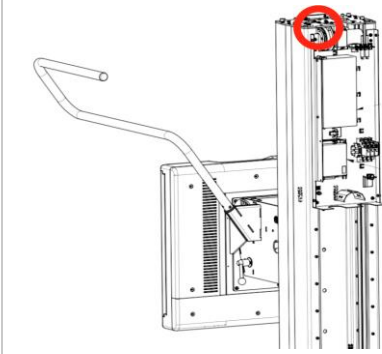
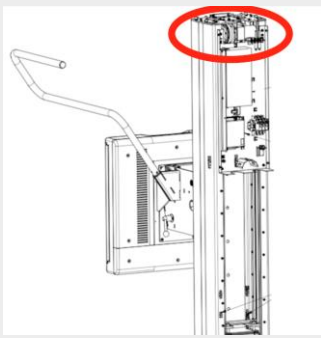
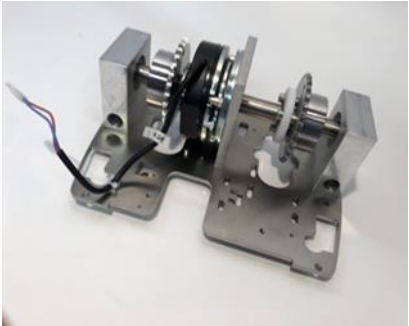
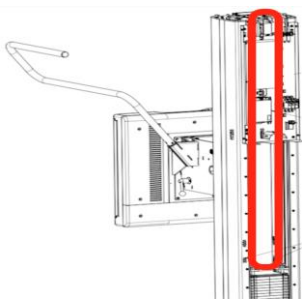
Wall stand (WS) and Wall stand Bucky

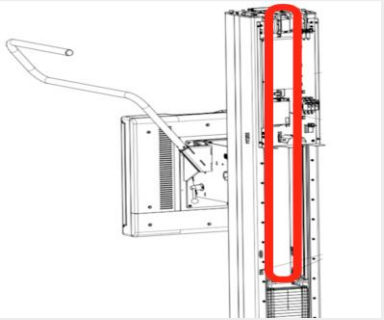
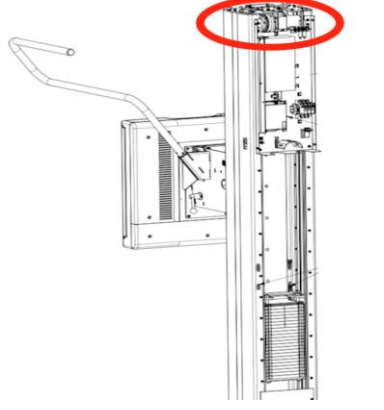
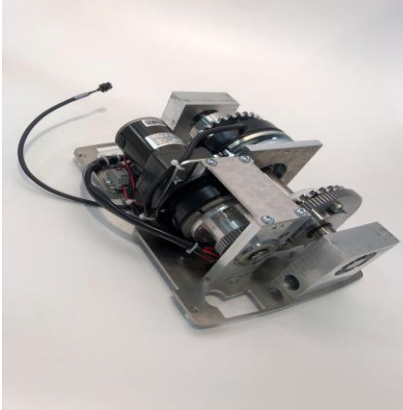
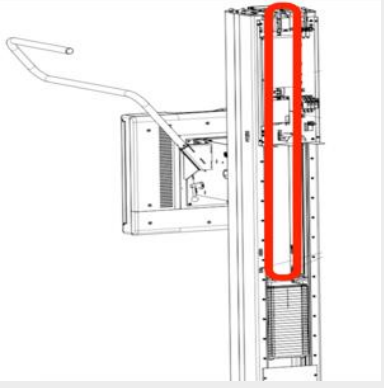
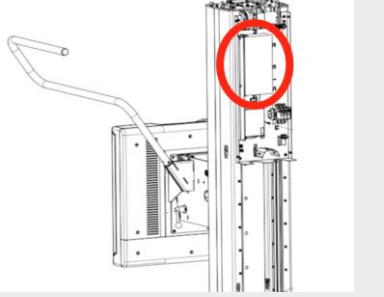


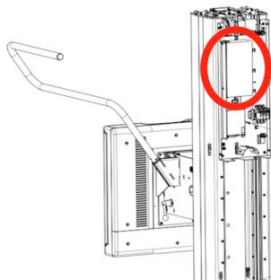
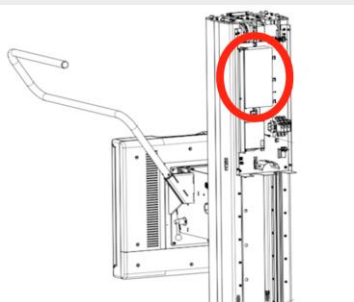
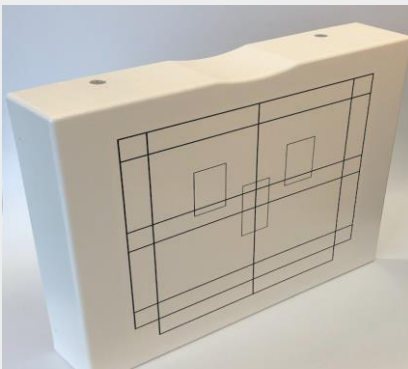
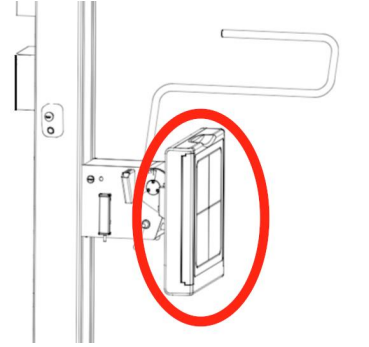
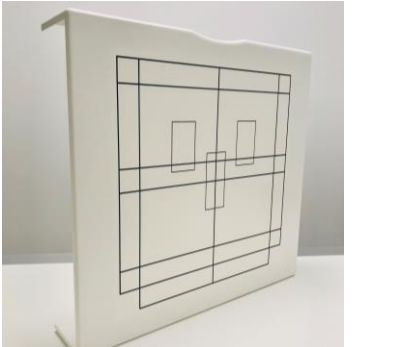
Description	Spare Part article no	Info/Number on spare part	Drawing	Photo
Emergency stop single 	0072-810-001	0175-750-020 Single stop = for WS without tilt		
Emergency stop dual 	0072-810-021	0175-750-021 Dual stop = for WS with tilt		
Tilt angulation label 	0175-810-110	0175-003-110 Left + Right		
Tilt levers Blue	0175-810-111	0072-626-110		
Tilt lever, WS Gray	0175-810-004	0072-629-110		
Foot control Z-brake release Manual vertical movement Blue	0175-810-007	0175-099-005		

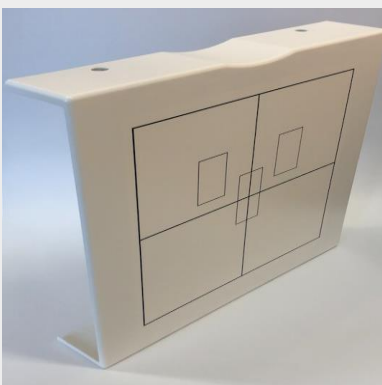
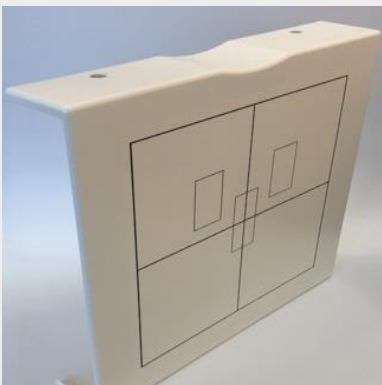
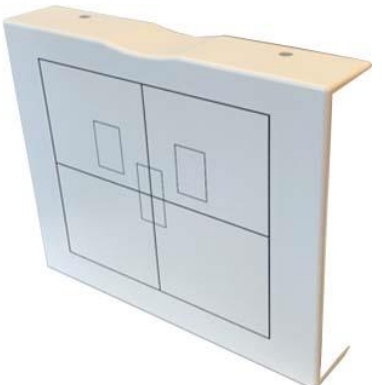
Foot control Z-brake release Manual vertical movement Grey	0072-810-011	0072-099-005		
Foot control WS, Manual and motorized vertical movements Blue	0175-810-008	0175-099-006 Has been replaced by 0072-810-012		
Foot control WS, Manual and motorized vertical movements Grey	0072-810-012	0175-099-006		
Control handle, collimation	0072-810-009	0072-099-009		
PCB CB/DC Z WS Master For motorized WS	0182-810-107	70-107 3.1CBDC01		

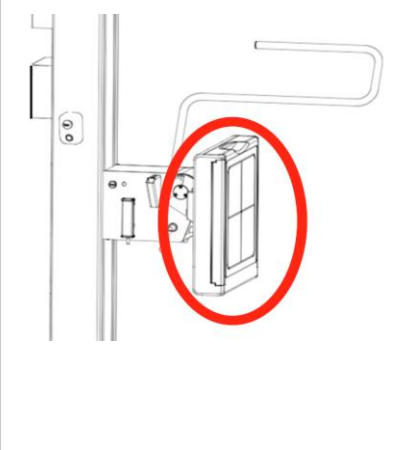
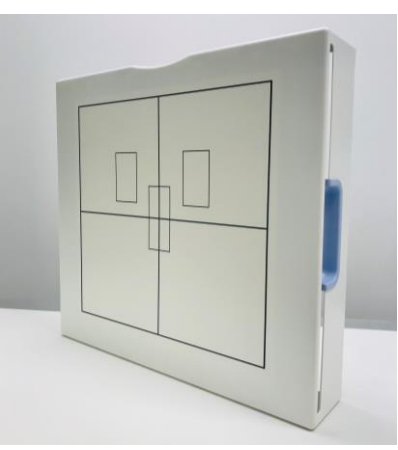
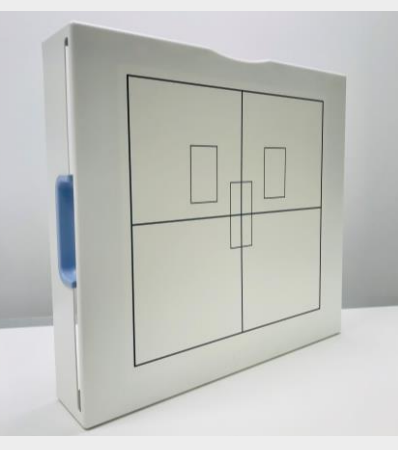
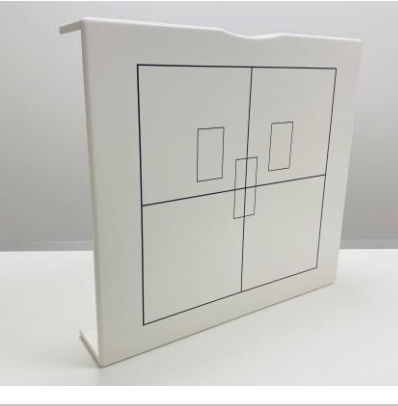
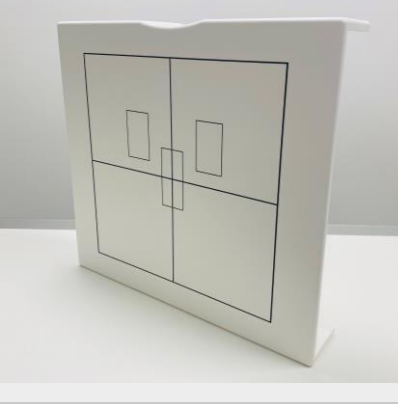
PCB CB Z WS Master For manual WS	0180-810-003	70-106 3.1CB01		
PCB CB WS SI	0180-810-004	70-106 3.1CB02		
Relay Omron 24VDC	0070-811-275	10-008 10-019 3.1RE01 3.1RE02 3.1RE03 3.1RE04		
Rubber end stops	0175-810-050	55-050 x2		
Gas spring, tilt 400 Nm	0175-810-117	51-117 51-152 x2 51-154 x2 M6M M8 nylock x2		

PCB PLB Indicator light	0175-810-610	72-610		
Absolute encoder WS	0512-810-012	0512-154-010 *Same encoder for Alpha in OTC		
Potentiometer Z, 10T 2K	0182-810-011	0182-099-011 Drawing: 0182-750-010		
Brake unit Z, For manual WS	0182-810-100	0182-154-100		
Chain including attachments, For manual WS with non-tilt bucky	0182-810-010	0182-154-010		

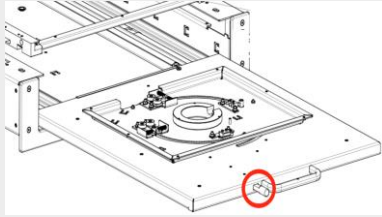
Chain including attachments, For manual WS with tilting bucky	0182-810-013	0182-154-012		
Motor unit for Z movement	0512-810-220	0512-099-020		
Chains and chain lock For motorized WS with/without tilting bucky	0512-810-020	0512-154-020		
Hose K1H 35mm RAL 9010 230cm	0200-810-019	05-019 Wall stand cable protection hose		
Power supply, For motorized WS only 24/36V	0512-810-409	12-409		

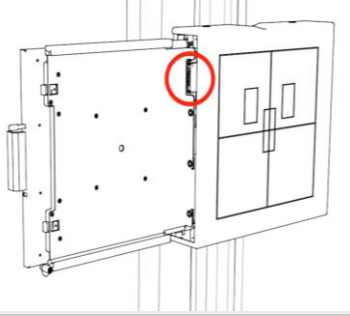
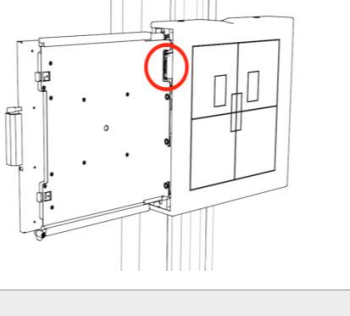
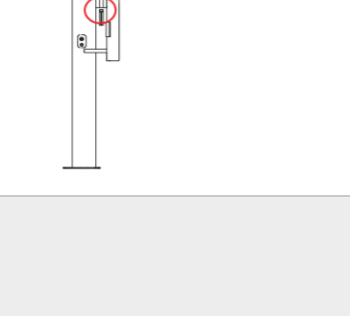
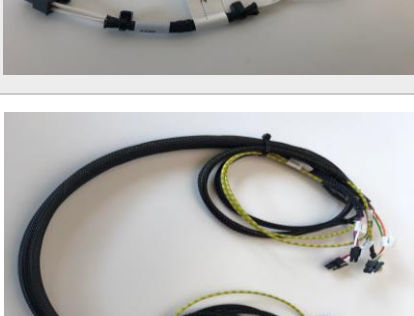
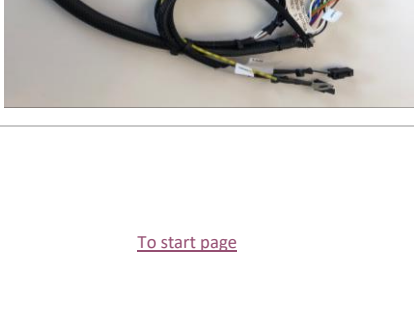
Power supply, 24V	0073-810-033	12-033		
Power supply, For motorized WS only 36V	0073-810-034	12-034		
AEC chamber, AID ICX3922B	1105-810-805	0200-154-003 90-805		
Cover, detector holder WS Right hand loading	0182-810-039	0182-154-039 0182-154-539 CXDI-701 CXDI-702 CXDI-703 CXDI-710 CDXI-720		
Cover, detector holder WS Left hand loading	0182-810-043	0182-154-043 0182-154-543 CXDI-701 CXDI-702 CXDI-703 CXDI-710 CDXI-720		

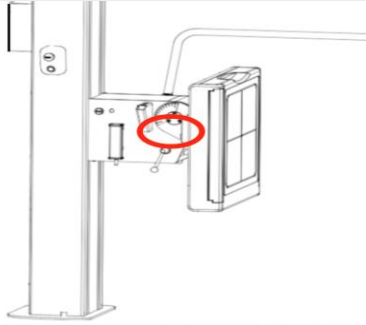
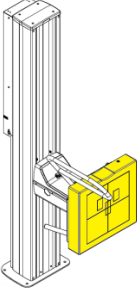
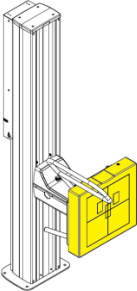
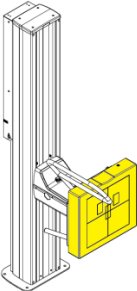
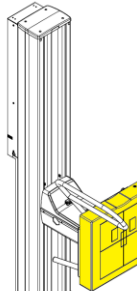
Cover, detector holder WS Left hand loading	0182-810-045	0182-154-045 0182-154-545 CXDI-401		
Cover, detector holder WS Right hand loading	0182-810-046	0182-154-046 0182-154-546 CXDI-401		
Cover, detector holder WS Left hand loading	0182-810-068	0182-154-068 0182-154-568 CXDI-402 CXDI-410		
Cover, detector holder WS Right hand loading	0182-810-055	0182-154-055 0182-154-555 CXDI-402 CXDI-410		

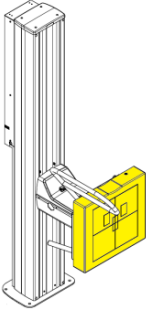
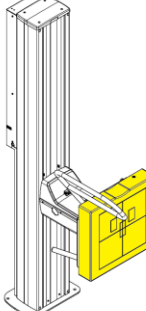
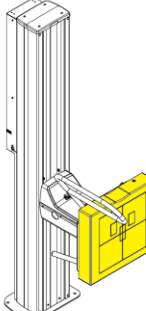
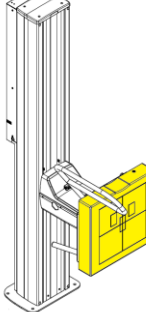
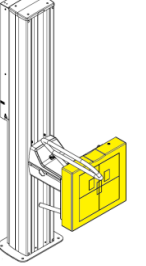
Cover, detector holder WS Right hand loading	0182-810-050	0182-154-057 0182-154-557 CXDI-401C Compact Stationary detector (Grid slot to the RIGHT)		
Cover, detector holder WS Left hand grid loading	0182-810-051	0182-154-056 0182-154-556 CXDI-401C Compact Stationary detector (Grid slot to the LEFT)		
Cover, detector holder WS Left hand loading	0182-810-041	0182-154-041 0182-154-541 CXDI-403C CXDI-420C		
Cover, detector holder WS Right hand loading	0182-810-049	0182-154-049 0182-154-549 CXDI-403C CXDI-420C		

Cover, detector holder WS Left hand grid loading		0182-154-058 0182-154-558 CXDI-420C Compact Stationary detector		
Cover, detector holder WS Right hand loading		0182-154-057 0182-154-557 CXDI-420C Compact Stationary detector		
Bearing shaft	0182-810-132	0182-017-132 MC6LS BN M5x10A MC6S M5x10		
Bearing	0182-810-126	0182-017-070 687 ZZ K6S M4x8A		
DH assy. Left	0182-810-124	For left hand loaded detector holders.		
DH assy. Right	0182-810-125	For right hand loaded detector holders.		

Button, bucky tray eject	0182-810-061	0182-017-061		
Screw, Index position. Grid	0180-810-051	55-107		
Cable bracket, detector For x10 and x02 series detectors.	0180-810-174	0180-017-174		
Cable bracket, detector For x20 and x03 series detectors.	0180-810-177	0180-017-177		
Cable rotation switch Portrait / Landscape	0182-810-005	0058-750-005 3.DSW04 3.DSW05	*Located underneath detector tray	
Cable top/center switches	0182-810-006	0058-750-006 3.DSW06 3.DSW07		

Cable Detector present switch	0182-810-037	0058-750-037 3.DSW08 3.DSW09		
Cable Detector present switch	0182-810-007	0058-750-012 3.DSW08		
Cable detector tray	0182-810-008	0058-750-017		
Button, Brake release Z WS	0182-810-015	0182-750-003		
Cable grid switch	0182-810-019	0058-750-019		
Main Cable for detector switches (size 14x17)	0182-810-020	0058-750-022 Main cable assembly connecting bucky unit with wall stand		

Main Cable for fixed detector (size 17x17)	0182-810-032	0058-750-032 <i>Only for grid switches: 3.DSW02 3.DSW03</i>		
Detector holder WS Left CXDI-402CW CXDI-410CW	0058-810-301	0058-151-301 Incl: pre-installed Detector cable. Excl: Cover p/n 0182-810-068 and AEC chamber p/n 1105-810-805.		
Detector holder WS Right CXDI-402CW CXDI-410CW	0058-810-302	0058-151-301 Incl: pre-installed Detector cable. Excl: Cover p/n 0182-810-055 and AEC chamber p/n 1105-810-805.		
Detector holder WS Left CXDI-403CW CXDI-420CW	0058-810-600	0058-151-600 Incl: pre-installed Detector cable. Excl: Cover p/n 0182-810-041 and AEC chamber p/n 1105-810-805.		
Detector holder WS Right CXDI-403CW CXDI-420CW	0058-810-601	0058-151-600 Incl: pre-installed Detector cable. Excl: Cover p/n 0182-810-049 and AEC chamber p/n 1105-810-805.		

Detector holder WS Left CXDI-703CW CXDI-720CW	0058-810-602	0058-151-601 Incl: pre-installed Detector cable. Excl: Cover p/n 0182-810-043 and AEC chamber p/n 1105-810-805.		
Detector holder WS Right CXDI-703CW CXDI-720CW	0058-810-603	0058-151-601 Incl: pre-installed Detector cable. Excl: Cover p/n 0182-810-039 and AEC chamber p/n 1105-810-805.		
Detector holder WS Left CXDI-702CW CXDI-710CW	0058-810-303	0058-151-300 Incl: pre-installed Detector cable. Excl: Cover p/n 0182-810-043 and AEC chamber p/n 1105-810-805.		
Detector holder WS Right CXDI-702CW CXDI-710CW	0058-810-304	0058-151-300 Incl: pre-installed Detector cable. Excl: Cover p/n 0182-810-039 and AEC chamber p/n 1105-810-805.		
Detector holder WS Left CXDI-401C Compact	0058-810-305	0058-151-303 Incl: pre-installed Detector cable. Excl: Cover p/n 0182-810-056 and AEC chamber p/n 1105-810-805.		

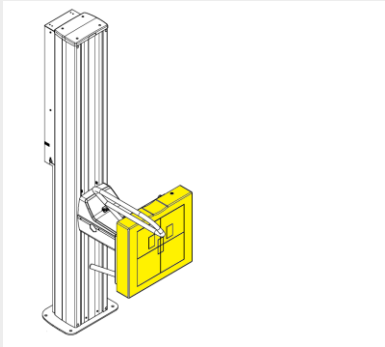
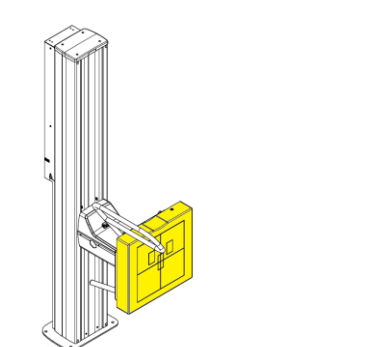
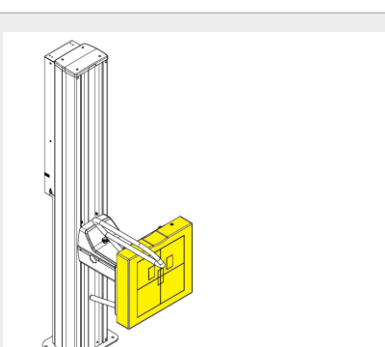
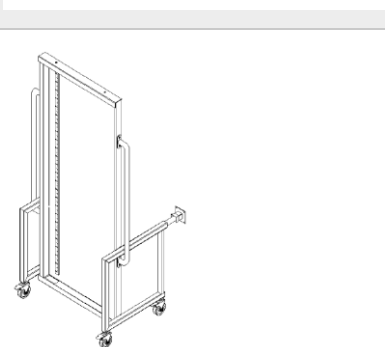
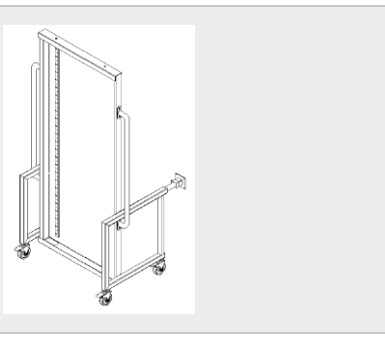
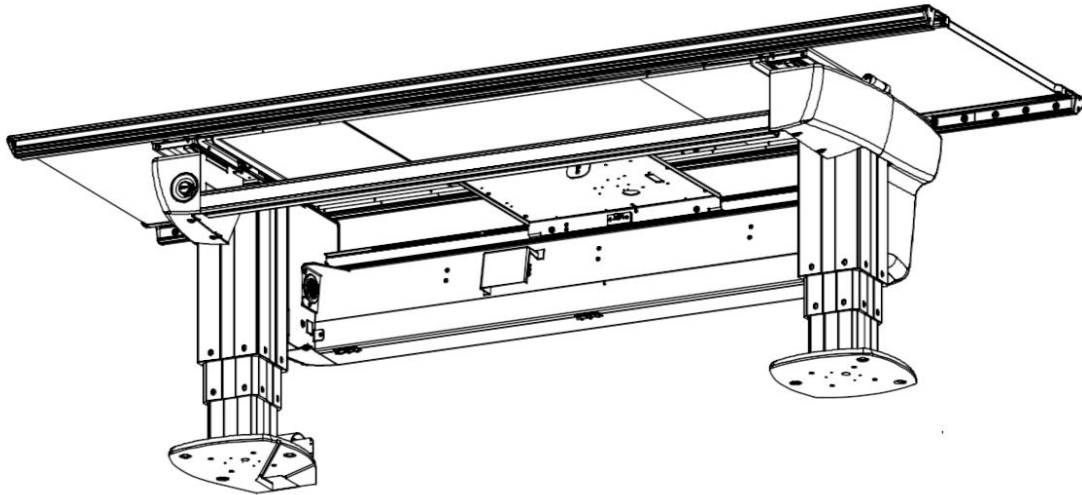
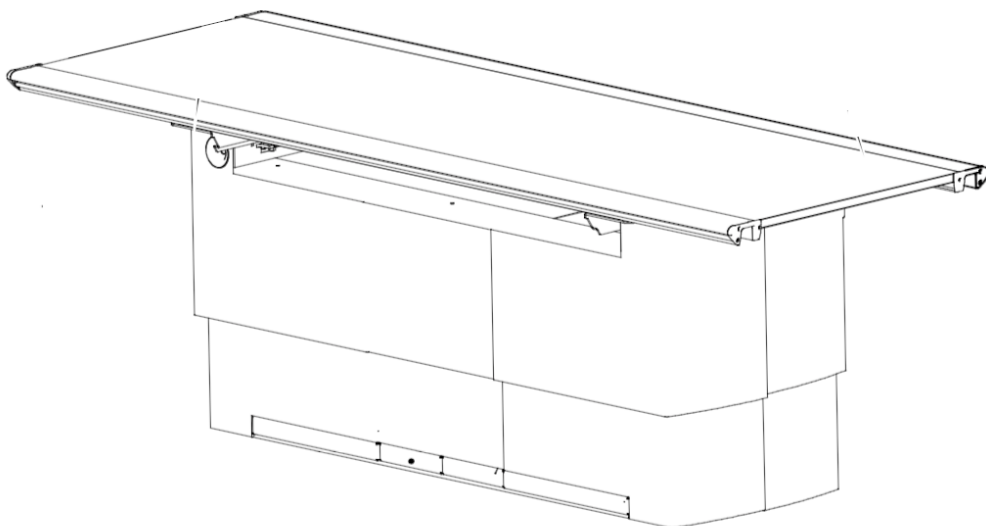
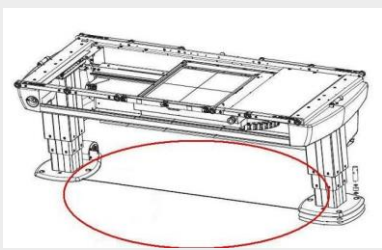
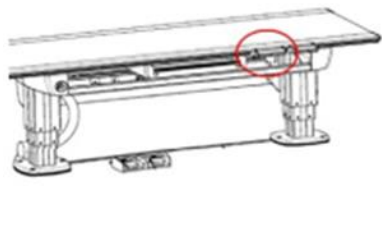
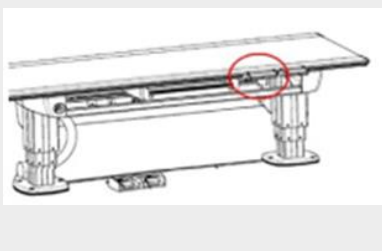
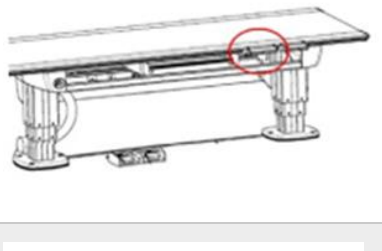
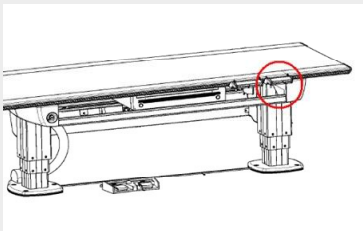
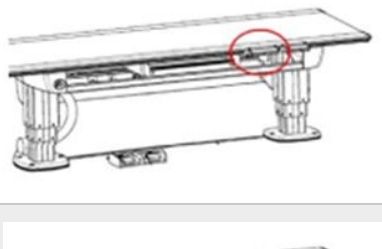
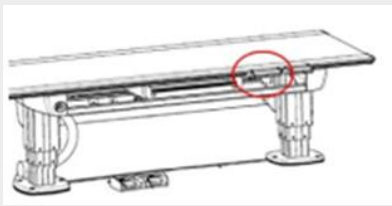
Detector holder WS Right CXDI-401C Compact	0058-810-306	0058-151-303 Incl: pre-installed Detector cable. Excl: Cover p/n 0182-810-057 and AEC chamber p/n 1105-810-805.		
Detector holder WS Left CXDI-420C Compact	0058-810-604	0058-151-603 Incl: pre-installed Detector cable. Excl: Cover p/n 0182-810-058 and AEC chamber p/n 1105-810-805.		
Detector holder WS Right CXDI-420C Compact	0058-810-605	0058-151-603 Incl: pre-installed Detector cable. Excl: Cover p/n 0182-810-057 and AEC chamber p/n 1105-810-805.		
Ruler 5mm 180cm Stitching stand 25-090	0072-810-094	068F00736		
Ruler fixation kit Stitching stand 25-090	0072-810-093	068F00728		

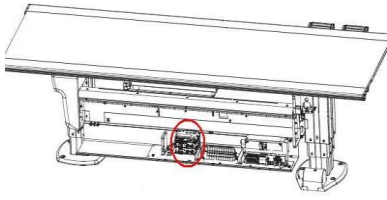
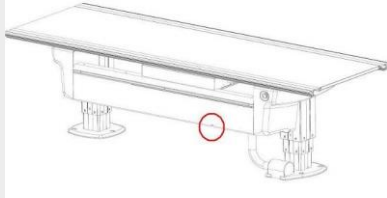
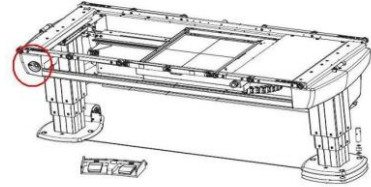
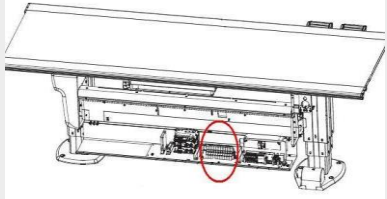
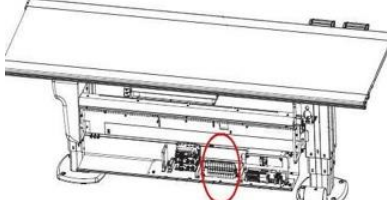
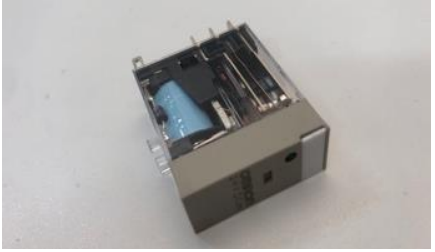
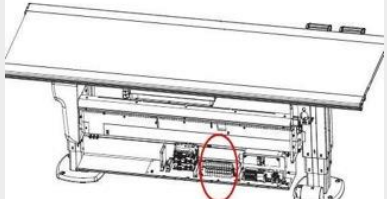
Table (TS) – Two colum

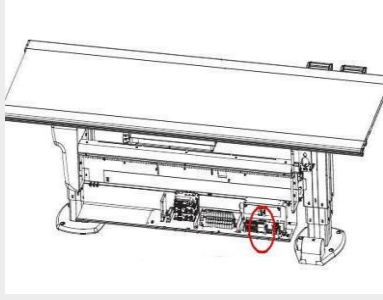
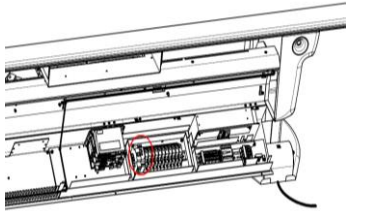
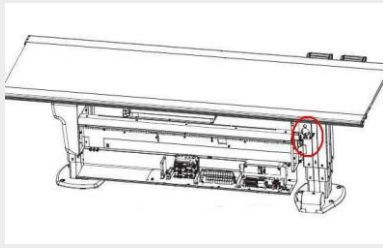
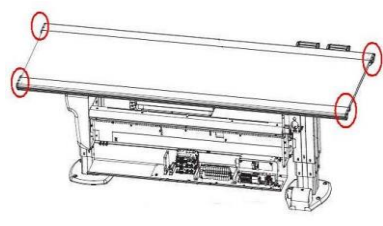
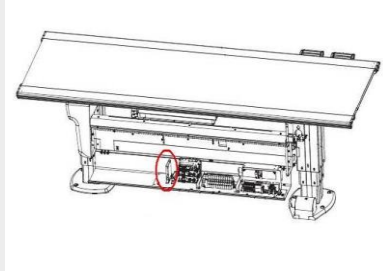
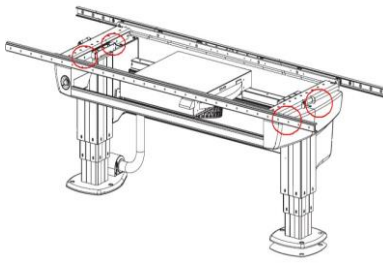


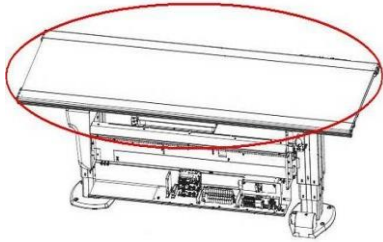
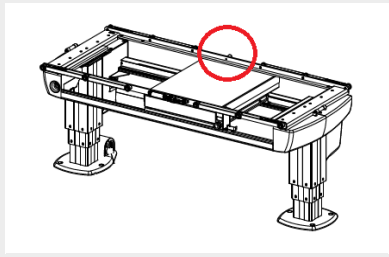
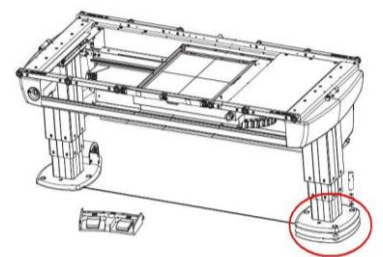
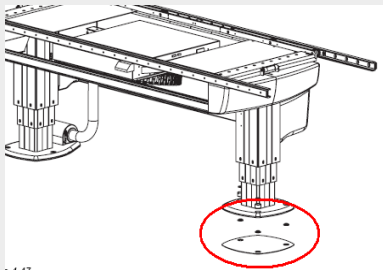
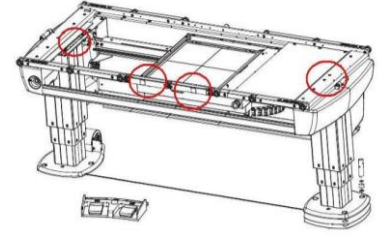
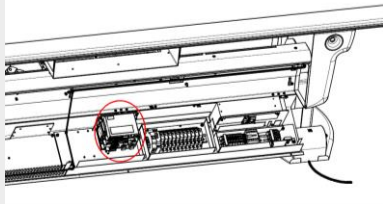
Some parts in this section are applicable also for the Closed table. See also separate section for Closed Table spare parts.

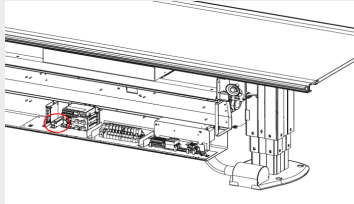
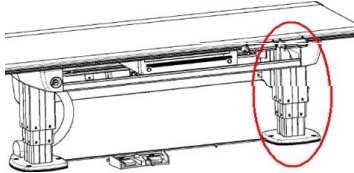
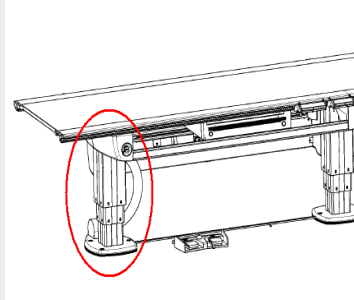
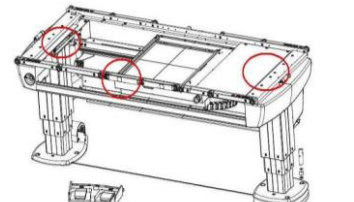
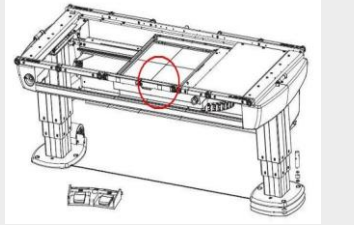
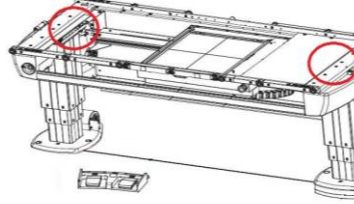


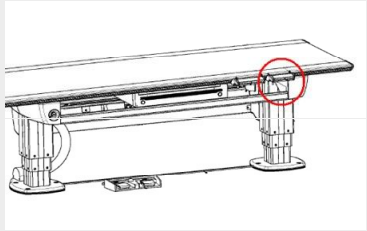
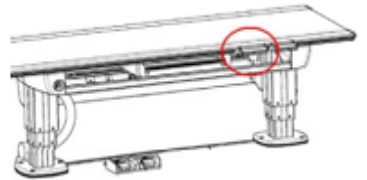
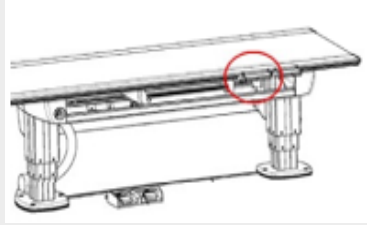
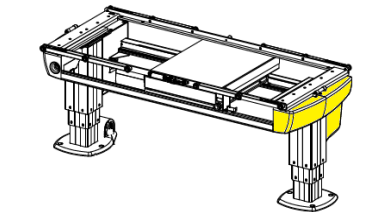
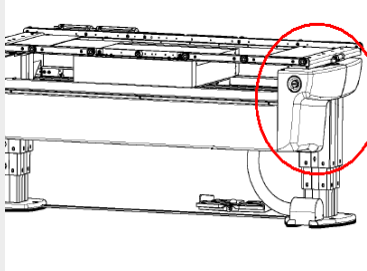
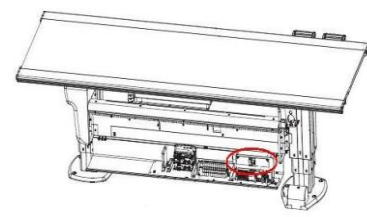
Description	Spare Part article no	Info/Number on spare part	Drawing	Photo
Foot switch strip X/Y	0055-810-005	0055-099-025 02-030 02-330		
Control handle, movements Blue 625	0055-810-008	0055-099-008 0055-625-124 0055-625-128 54-064 x2		
Control handle, movements Gray 629		0055-099-008 0055-629-124 0055-629-128 54-064 x2		
Control handle, movements White 660		0055-099-008 0055-660-124 0055-660-128 54-064 x2		
Control handle, Collimation Blue 625	0055-810-009	0181-099-009 0055-625-124 0055-625-128 54-064 x2		
Control handle, Collimation Gray 629	0055-810-018	0181-099-009 0055-629-124 0055-629-128 54-064 x2		
Control handle, Collimation White 660		0181-099-009 0055-660-124 0055-660-128 54-064 x2		

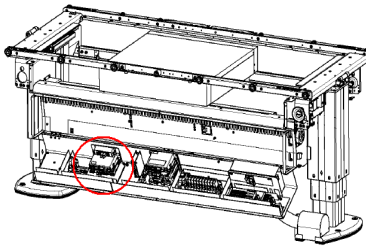
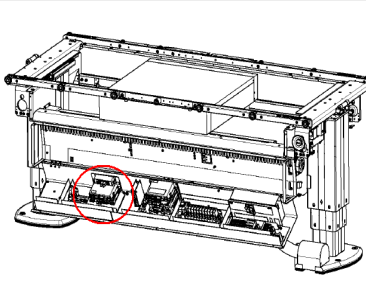
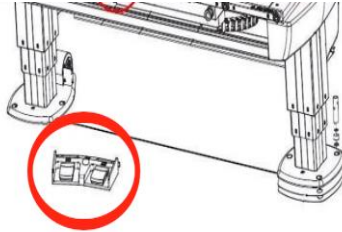
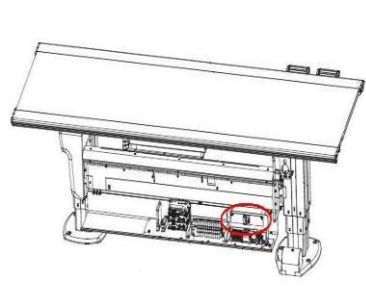
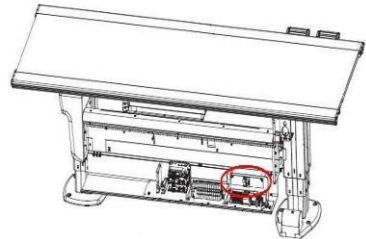
PCB CB/DC 20 Master Z	0055-810-010	70-104 0048-750-110		
Signal lamp	0055-810-014	19-025		
Emergency stop	0055-810-016	02-283 02-208 02-212 + kabel		
Relays, Omron (11 pcs)	0055-810-020	10-019		
Relay, 24VDC, 2pol Coil	1105-810-071	10-019 2.1RE03, 2.1RE04 2.1RE05, 2.1RE06 2.1RE07, 2.1RE08 2.1RE09, 2.1RE10 2.1RE11, 2.1RE12 2.1RE13, 2.1RE14		
Relay 24VDC. 2pol Socket	1105-810-072	10-008		

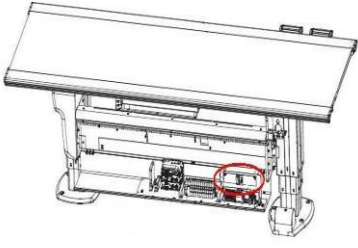
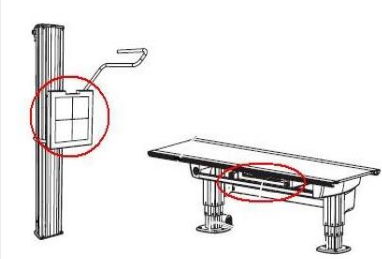
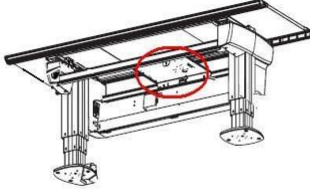
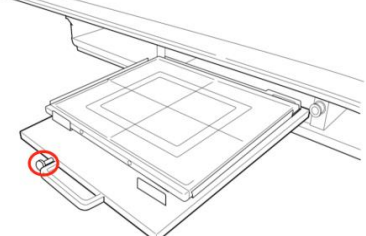
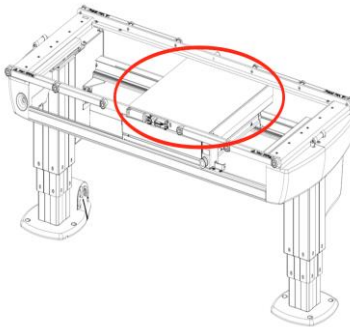
Relay, Emergency stop	0055-810-025	10-053 2.1RE01		
Relay, Watchdog	0500-810-996	10-060		
Tilt sensor	0055-810-030	20-050		
Table top end covers	0055-810-031	0055-009-030 0055-009-031 4pcs		
PCB Guard board kit	0055-810-035	0055-650-786 MC6S M4x6A		
Load cell, guard kit	0055-810-045	20-055 02-607 02-608 + washers		

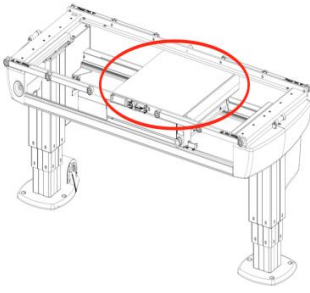
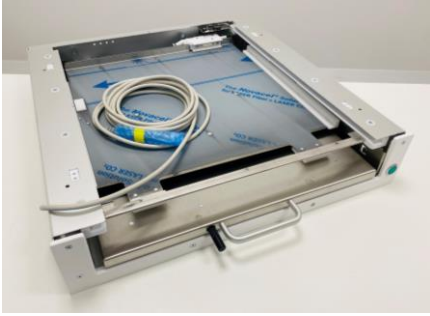
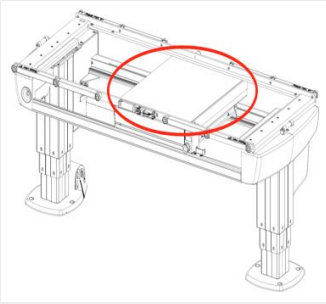
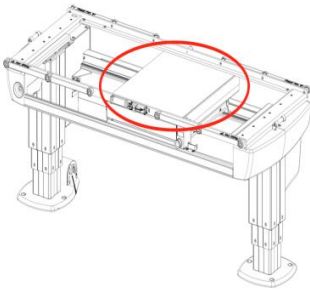
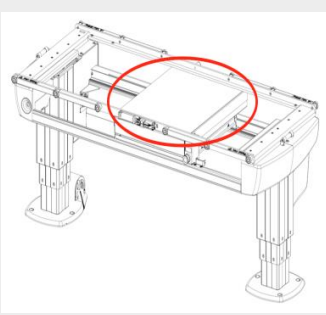
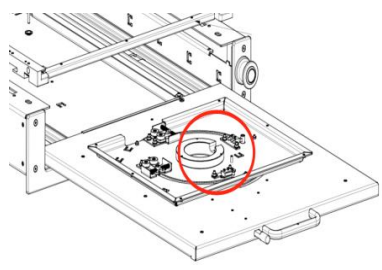
Tabletop Eq: 0,9mm Al	0055-810-041	0055-099-041		
Mechanical endstop X	0055-810-323	0055-006-323 02-129 MF6S M8x45A		
Insulation kit for foot plate	0055-810-060	0055-008-105 0055-008-106 x6 0055-008-107 x6 0055-008-205		
Schims for foot plate	0055-810-065	0055-900-035 x5 0055-900-036 x5 0055-900-037 x3 0055-900-038 x3		
Cable kit brake X/Y	0055-810-085	0055-750-080 0055-750-085		
Fuse 15A (5 pcs)	0055-810-090	09-017		

Bcm box complete	0055-810-110	0055-099-110		
Table column Foot/Right side	0055-810-201	0055-605-101		
Table column Head/Left side	0055-810-205	0055-605-105		
Tabletop brake unit, left side	0055-810-552	0055-002-552		
Tabletop brake unit, right side	0055-810-553	0055-002-553		
Brake unit, tabletop, Y Used in Y direction.	0055-810-490	0055-003-490		

Handlebar side covers Blue 625	0072-810-005	0055-625-124 0055-625-128 54-064 x2		
Side covers, handlebar, kit Gray 629	0072-810-010	0055-629-124 0055-629-128 54-064 x2		
Side covers, handlebar, kit White 660	0055-810-109	0055-660-124 0055-660-128 54-064 x2		
Table covers Foot/Right side	0055-810-750	0055-605-055 0055-605-655		
Table covers Head/Left side	0055-810-755	0055-605-066 0055-605-070		
PCB OVP Over voltage protection	0055-815-760	0055-154-760		

PCB CB Table SI Node	0180-810-006	70-106 2.1CB02		
PCB CB Table Master node	0180-810-005	70-106 2.1CB01		
Foot control X/Y/Z	0055-810-004	0072-099-004		
Power supply 24/36V	0512-810-409	12-409		
Power supply 36V	0073-810-034	12-034		

Power supply 24V	0073-810-033	12-033		
AEC chamber, AID	1105-810-805	0200-154-003		
Microswitch for bucky	0512-810-201	0512-750-201		
Button, bucky tray eject	0182-810-061	0182-017-061		
Cover detector holder	0512-810-100	0180-154-100		

Detector holder Table CXDI-402 CXDI-410	0058-810-200	0058-151-201 Incl: pre-installed Detector cable. Excl: Cover p/n 0181-810-100 and AEC chamber p/n 1105-810-805.		
Detector holder Table CXDI-403 CXDI-420				
Detector holder Table CXDI-702 CXDI-710	0058-810-201	0058-151-200 Incl: pre-installed Detector cable. Excl: Cover p/n 0181-810-100 and AEC chamber p/n 1105-810-805.		
Detector holder Table CXDI-703 CXDI-720				
Cable rotation Portrait/Landscape switches	0181-810-116	0058-750-016 2.DSW02 2.DSW03		

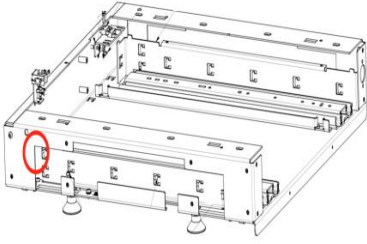
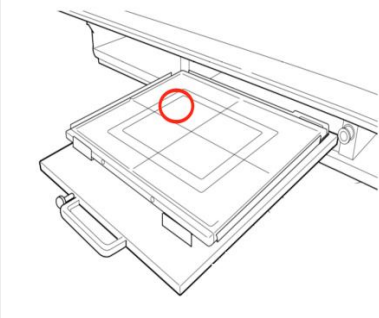
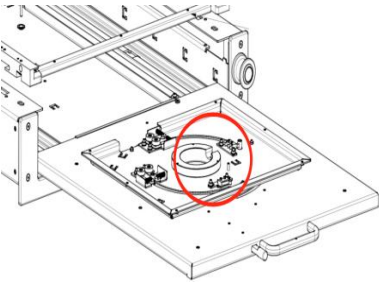
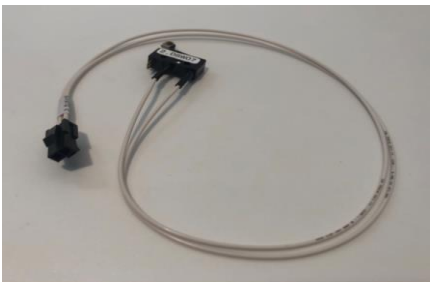
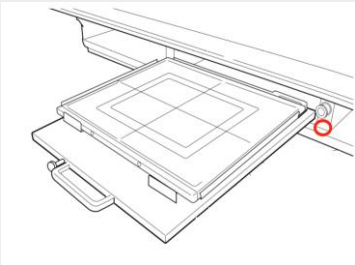
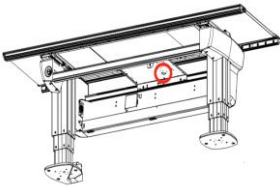
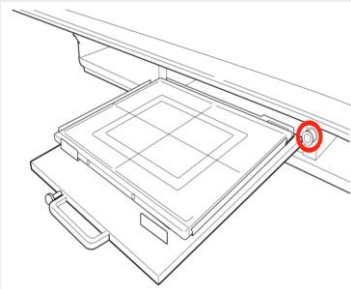
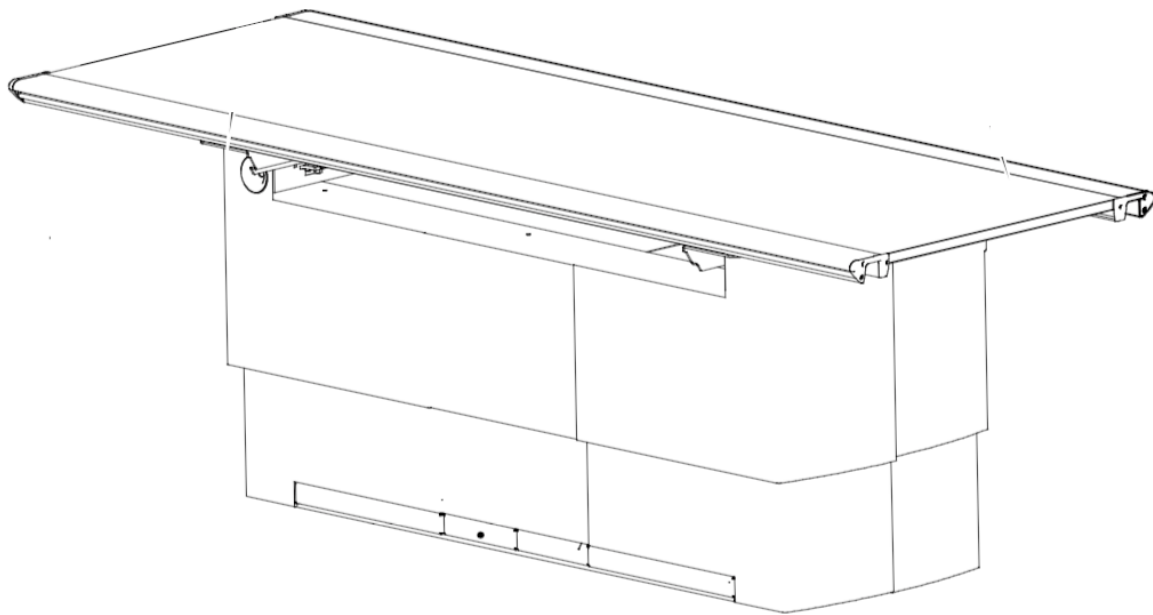
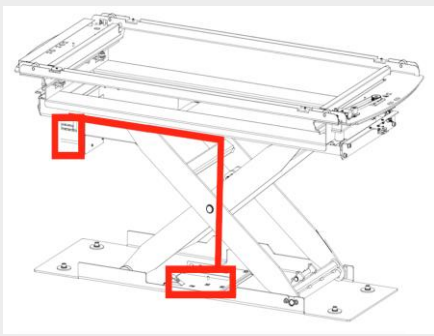
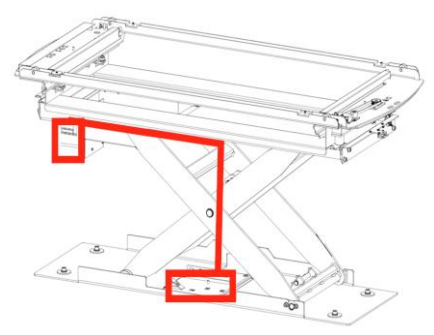
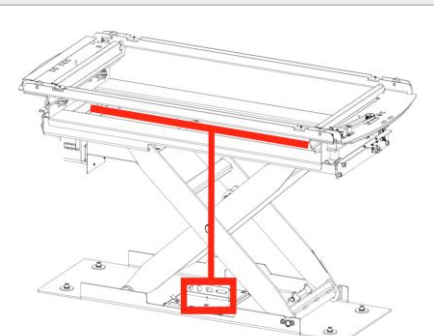
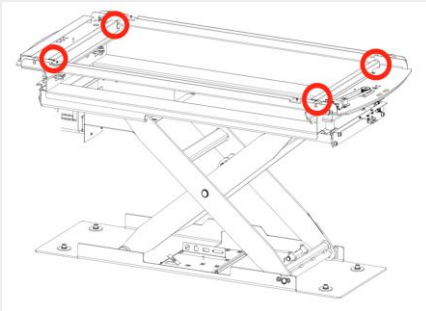
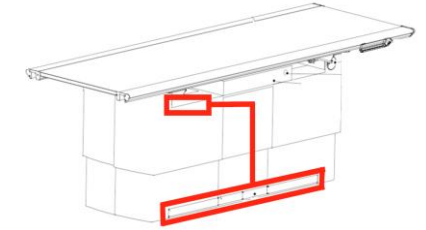
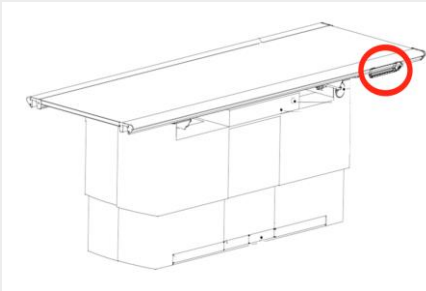
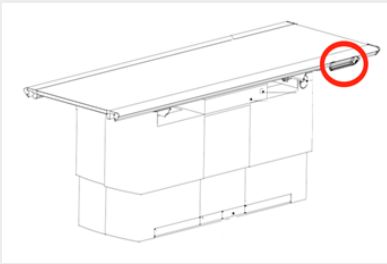
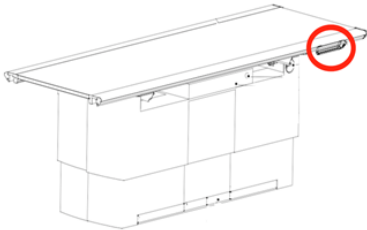
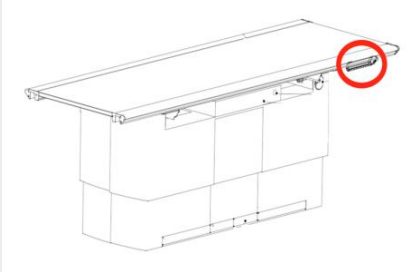
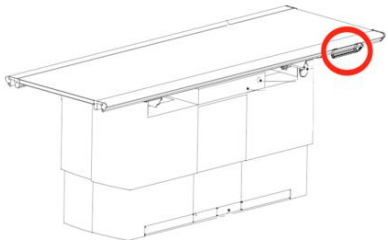
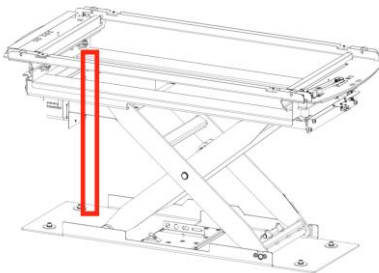
Detector tray switch	0181-810-107	0058-750-008 <i>2.DSW06</i>		
Detector present switch	0181-810-129	0058-750-029 <i>2.DSW06</i>		
Detector size switch	0181-810-138	0058-750-038 <i>2.DSW07</i>		
Grid present LED (green)	0181-810-104	0181-750-104 <i>2.DD01</i>		
Cable bucky	0181-810-109	0058-750-009		
Indication light/Release button	0181-810-066	0181-750-028		

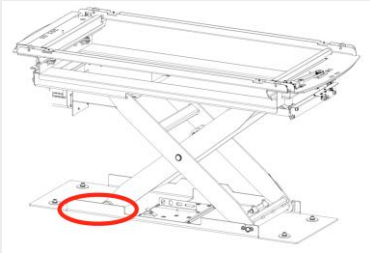
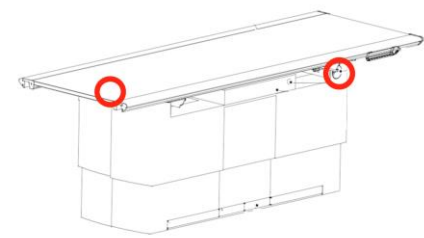
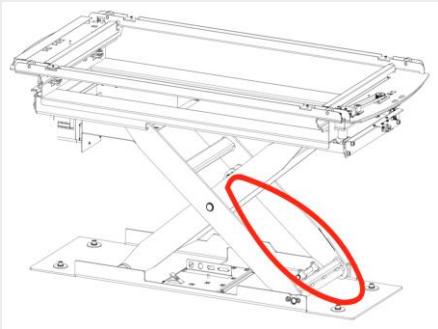
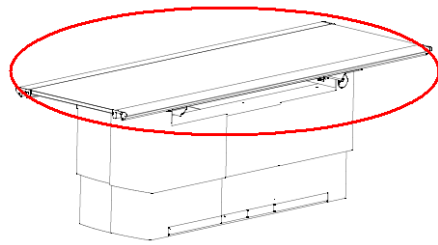
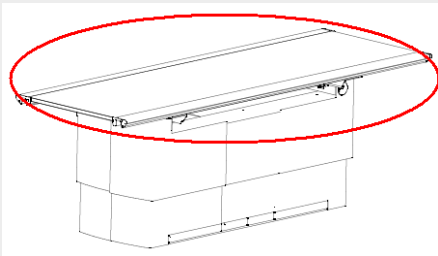
Table (TS) - Closed

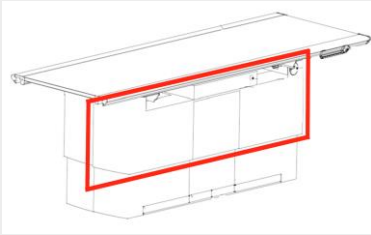
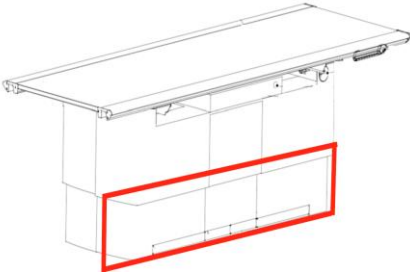
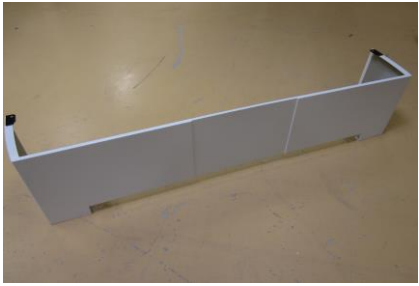
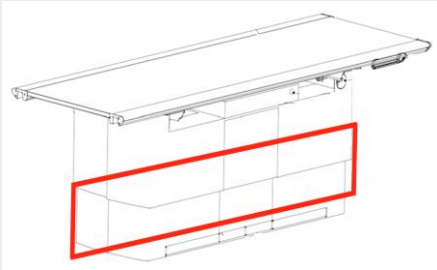
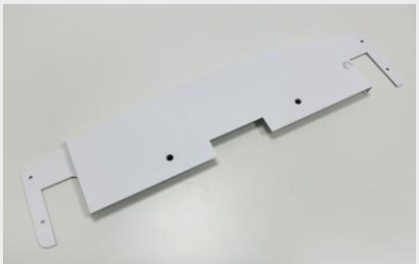
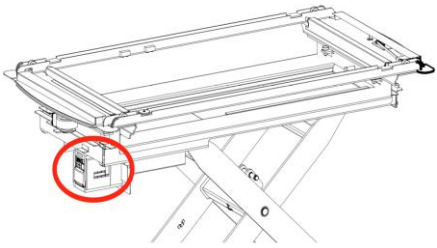


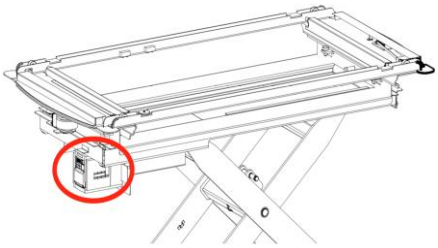
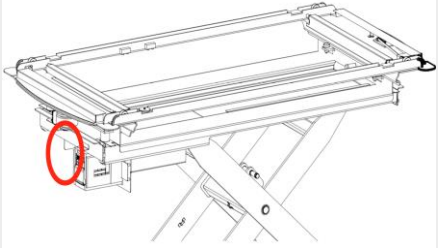
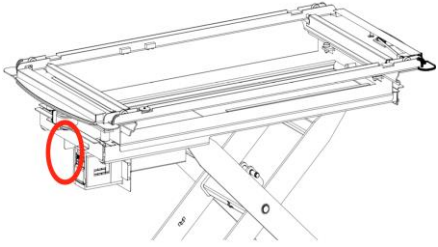
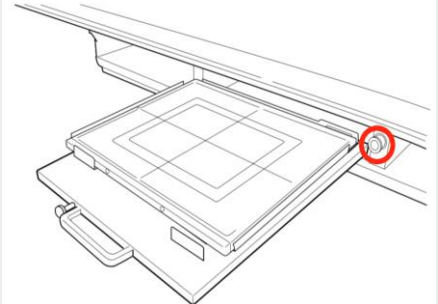
Description	Spare Part article no	Info/Number on spare part	Drawing	Photo
Cable Z kit	0181-810-003	0181-750-003 <i>Cable 2.1 to 2.2</i>		
Cable Z kit	0181-810-141	0181-750-041 <i>Cable 2.1 to 2.2</i>		
Cable 2.2 to 2.2FRE01	0181-810-212	0181-750-012		
Cable kit, Detector holder	0181-810-006	0181-750-006 <i>Cable 2.2 to 2.D</i>		
Foot control XYZ	0181-810-004	0181-099-004		

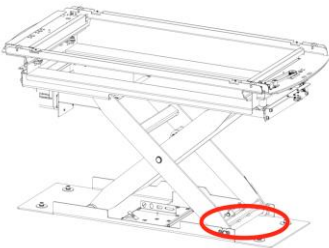
Foot control XYZ Gray	0055-810-019	0072-099-004		
Tilt stop, table	0180-810-123	0180-001-123		
Cable kit, Guard function	0181-810-007	0181-750-007		
Cable kit, Kick box	0181-810-015	0181-750-015		
Manoeuvre handle, XYZ Blue 625	0181-810-008	0181-099-008 0055-625-124 0055-625-128 54-064 x2		

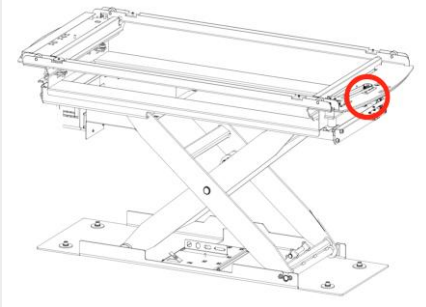
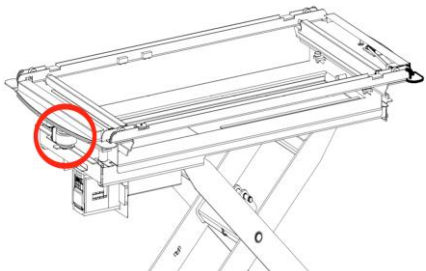
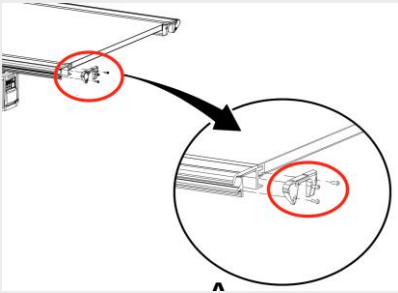
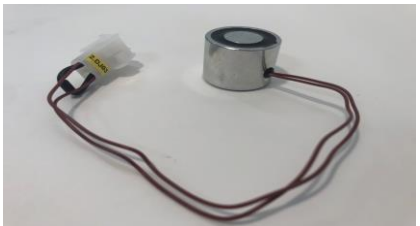
Manoeuvre handle, XYZ Gray 629	0181-810-013	0181-099-008 0055-629-124 0055-629-128 54-064 x2		
Manoeuvre handle, XYZ White 660	0181-810-011	0181-099-008 0055-660-124 0055-660-128 54-064 x2		
Manoeuvre handle, Collimation Blue 625	0181-810-009	0181-099-009 0055-625-124 0055-625-128 54-064 x2		
Manoeuvre handle, Collimation Gray 629		0181-099-009 0055-629-124 0055-629-128 54-064 x2		
Manoeuvre handle, Collimation White 660		0181-099-009 0055-660-124 0055-660-128 54-064 x2		
Potentiometer Z 10 kOhms	0181-810-010	0181-099-010		

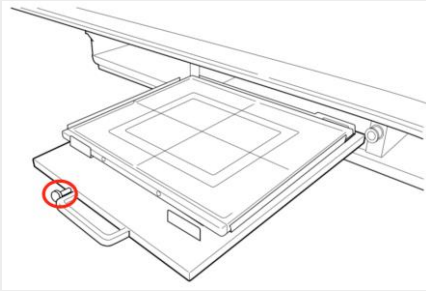
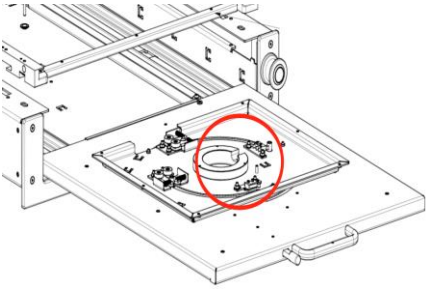
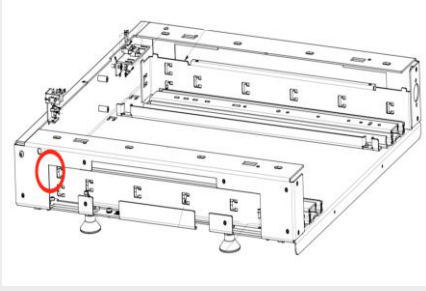
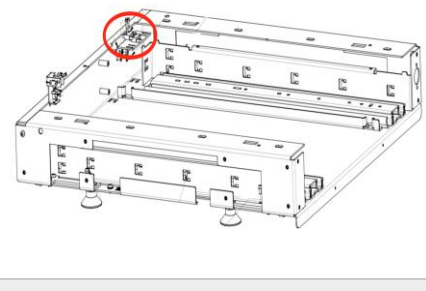
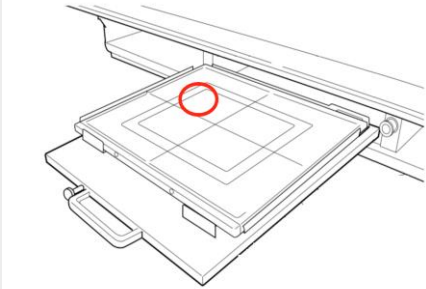
Cable kit, End stop Z	0181-810-016	0181-750-016		
Cable kit, Emergency stop	0181-810-018	0181-750-018		
Drive unit Z	0181-810-020	0180-099-020		
Table top	0181-810-044	0181-151-055		
Mechanical end stop Table top	0181-810-045	2x 0180-002-213 2x 55-029 6x K6S M4x20		

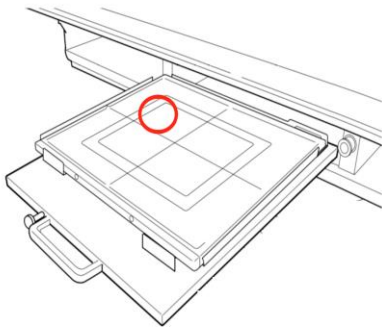
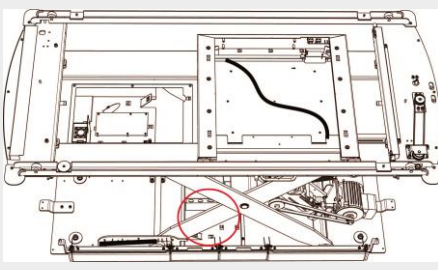
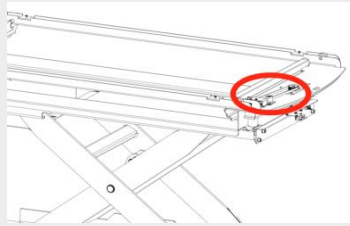
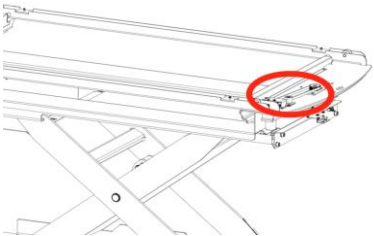
Cover, Upper front/back	0181-810-040	0180-605-040		
Cover, Front lower	0181-810-050	0180-605-050		
Cover, Back lower	0181-810-055	0180-605-055		
Edge strips, Table covers	0181-810-056	22000177 22000178 2pcs 375mm 2pcs 425mm		
Side cover	0180-810-060	0180-605-060		
Frequency converter MX2	0181-810-063	13-063 2.1FRE01		

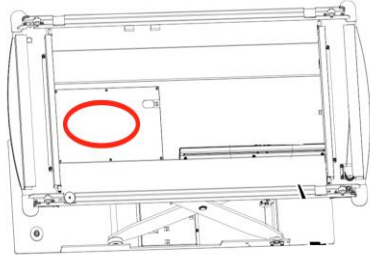
Frequency converter 3G3MX2-V2	0181-810-064	13-064 2.1FRE01 Used from s/n 80100.		
PCB CB Table SI Node	0180-810-006	70-106 2.1CB02		
PCB CB Table Master node	0180-810-005	70-106 2.1CB01		
Button, Brake Release Detector Holder	0181-810-066	0181-750-028		
Cover, detector holder	0181-810-100	0180-154-100		

Detector holder Table CXDI-402 CXDI-410	0058-810-400	0058-151-400 Incl: pre-installed Detector cable. Excl: Cover p/n 0181-810-100 and AEC chamber p/n 1105-810-805.		
Detector holder Table CXDI-403 CXDI-420	0058-810-701	0058-151-701 Incl: pre-installed Detector cable. Excl: Cover p/n 0181-810-100 and AEC chamber p/n 1105-810-805.		
Detector holder Table CXDI-702 CXDI-710	0058-810-401	0058-151-401 Incl: pre-installed Detector cable. Excl: Cover p/n 0181-810-100 and AEC chamber p/n 1105-810-805.		
Detector holder Table CXDI-703 CXDI-720	0058-810-700	0058-151-700 Incl: pre-installed Detector cable. Excl: Cover p/n 0181-810-100 and AEC chamber p/n 1105-810-805.		
Drive Belt Z	0181-810-102	52-191 PJ889		

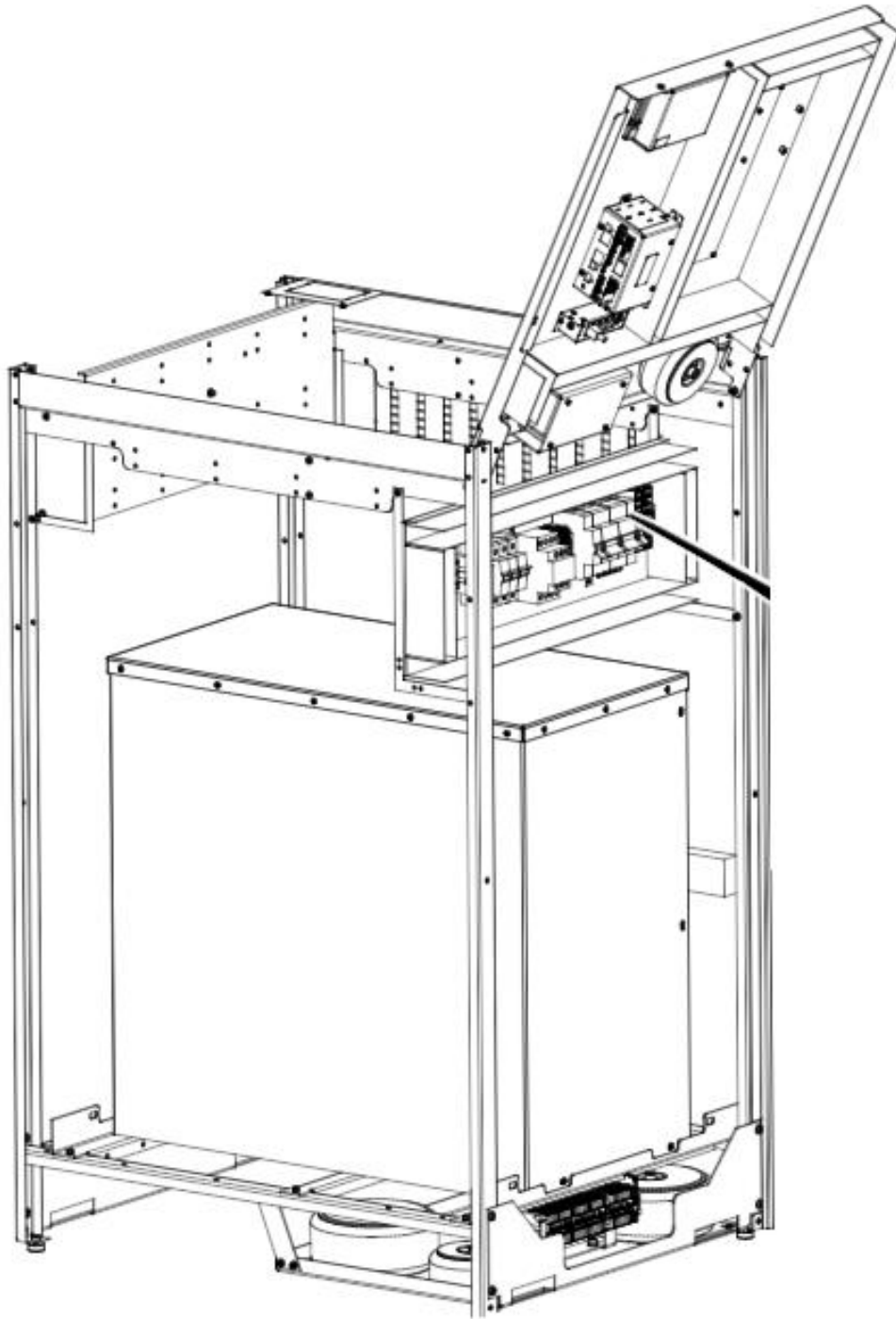
Brake Unit, X/Y	0181-810-110	0180-099-010 2.3BRA01 2.3BRA02		
Brake unit, Y	0181-810-115	0180-099-015 2.4BRA01		
Cable 2.2 to 2.3 Tabletop brake cable	0181-810-304	0181-750-004 Cable, TCB J18 to 2.3BRA01/02 (LONG/LAT)		
Cable 2.2 to 2.3 Tabletop brake cable	0181-810-305	0181-750-005 Cable, TCB J17 to 2.4BRA01 (LAT)		
End covers, tabletop (4 pcs, left/right)	0181-810-176	0181-001-112 0181-001-113		
Magnet unit, Detector Holder	0181-810-204	0181-750-027		

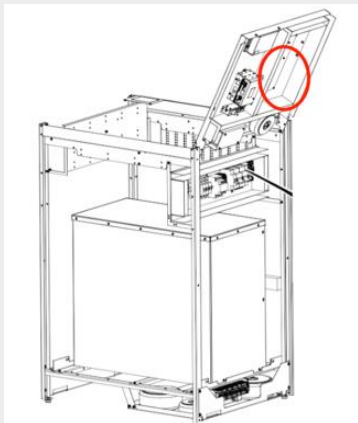
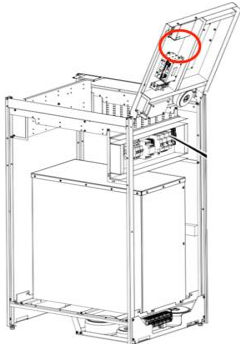
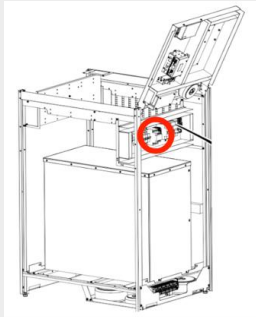
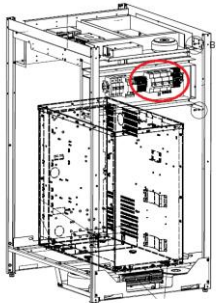
Button, bucky tray eject	0182-810-061	0182-017-061		
Cable rotation Portrait/ Landscape switches	0181-810-116	0058-750-016 2.DSW02 2.DSW03		
Detector tray switch	0181-810-108	0058-750-008 2.DSW06		
Cable grid switches	0181-810-118	0058-750-018 2.DSW04 2.DSW05		
Detector present switch	0181-810-129	0058-750-029 2.DSW06		

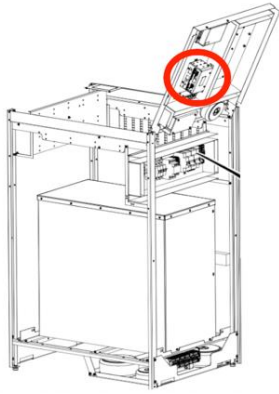
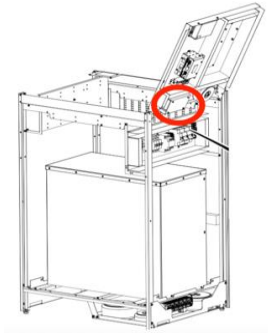
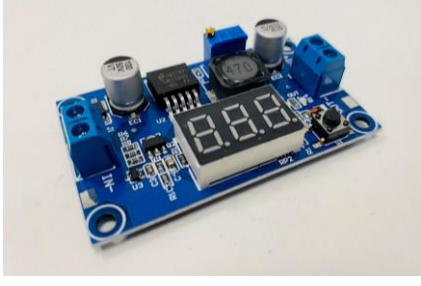
Detector size switch	0181-810-138	0058-750-038 <i>2.DSW07</i>		
Cable, SC to 2.1 System Cabinet to Table	0181-810-140	0181-750-040		
Cable bucky	0181-810-109	0058-750-009 <i>Cables routed from moving bucky to table base</i>		
Tooth belt, Brake X	0181-810-301	0180-001-301 <i>AT5/600 8mm</i>		
Housing AT5 Table top brake X	0180-810-340	0180-001-340		

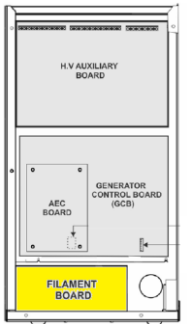
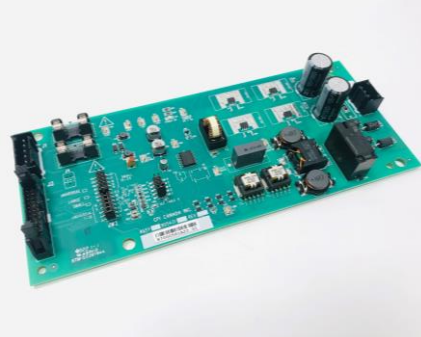
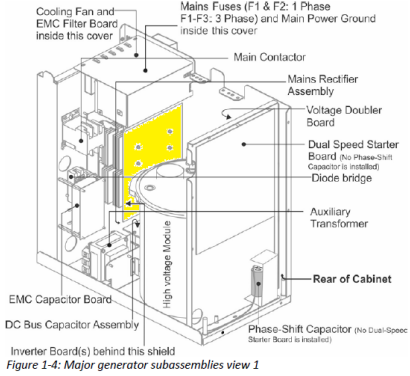
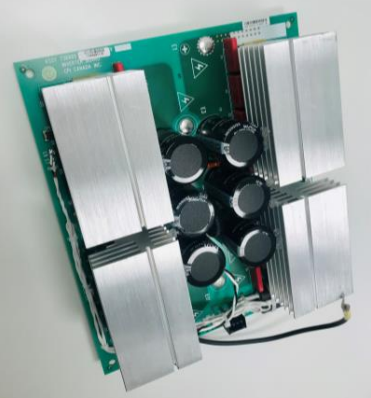
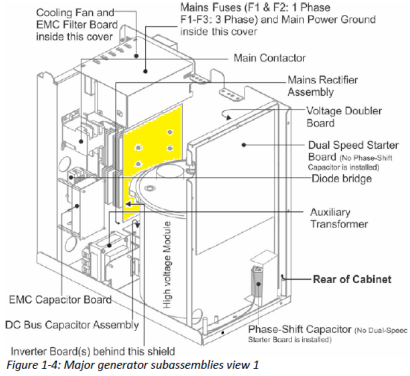
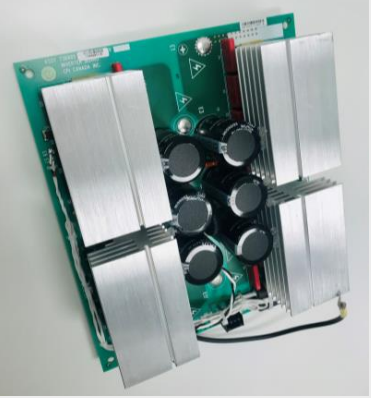
PCB TCB	0181-810-353	71-353 72-353		
Rail, Rollco 3080-500	0055-810-213	53-213 (1pc)		
Soft stop bumper Detector holder tray	1105-810-213	58455 (2pcs)		

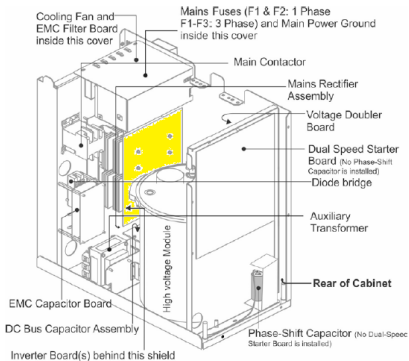
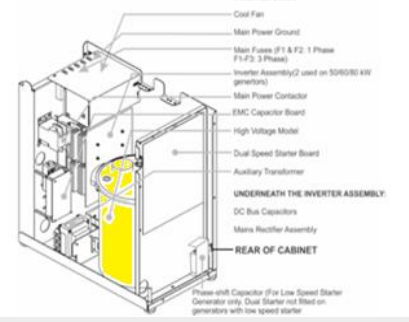
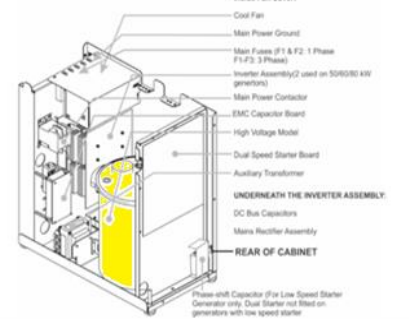
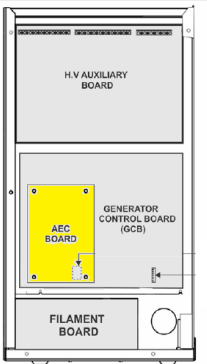
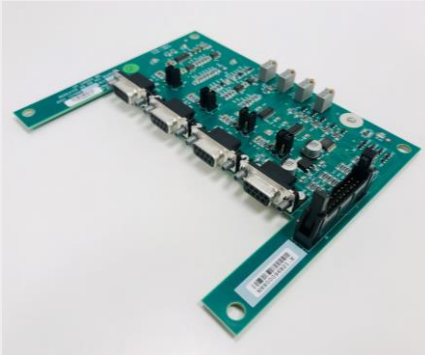
System Cabinet (SC)

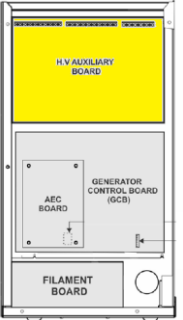
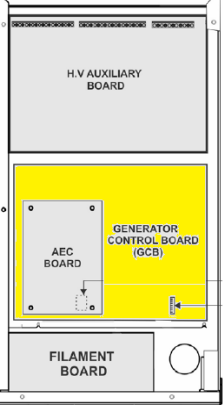
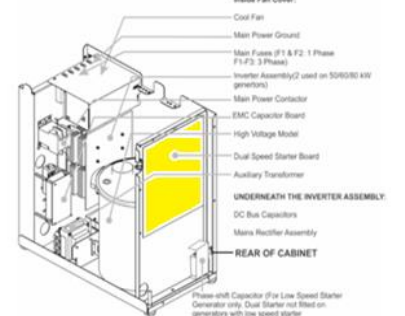
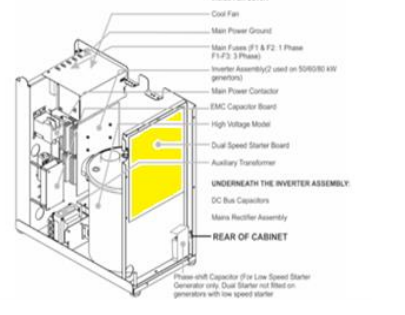
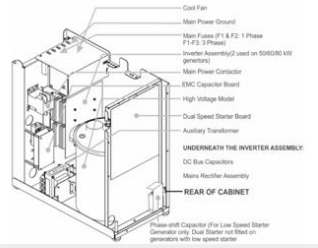


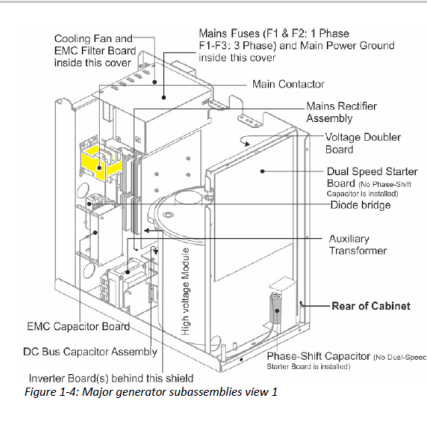
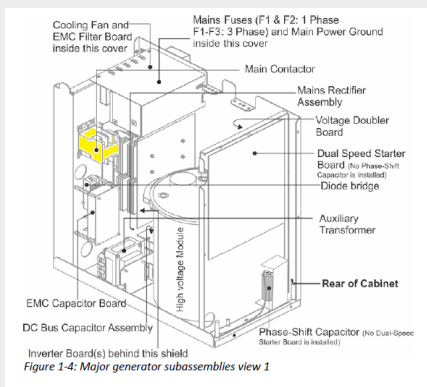
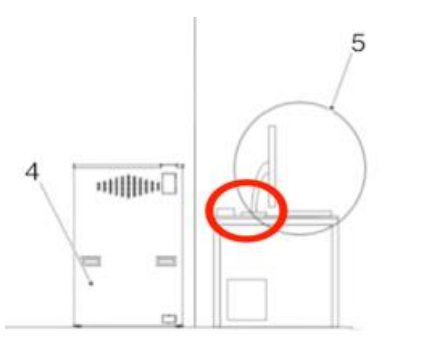
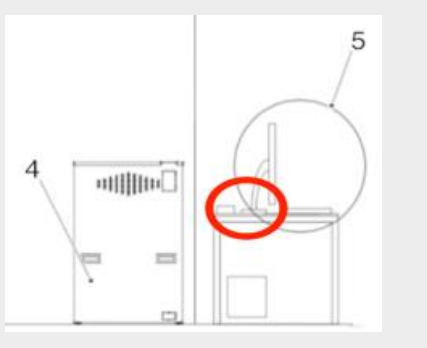
Description	Spare Part article no	Info/Number on spare part	Drawing	Photo
PCB FIB	0073-810-608	72-608 4.4FIB01		
PCB ESI	0184-810-240	71-240 72-240 4.4ESI01		
Relay 4.2RE01 (20A)	0170-810-056	10-056		
Circuit breaker 4.2F04, 6A C60N2PC6A	0170-810-197	09-097		
PCB CB800	0180-810-001	70-201		

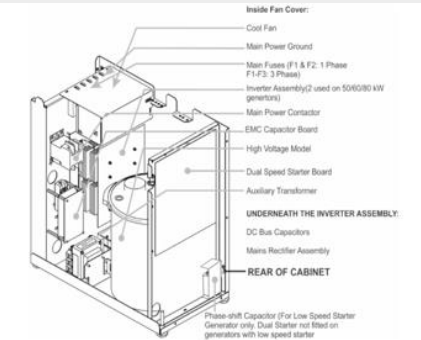
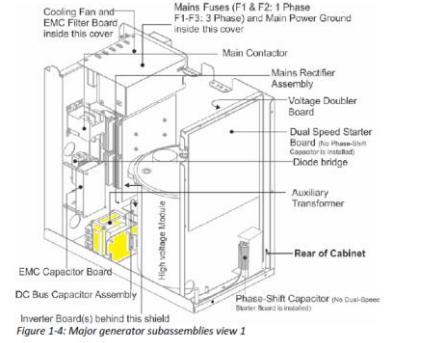
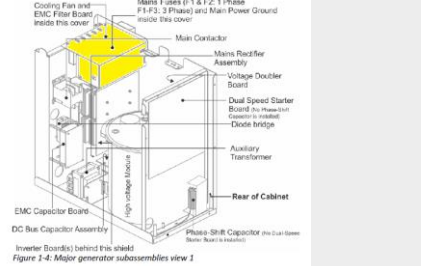
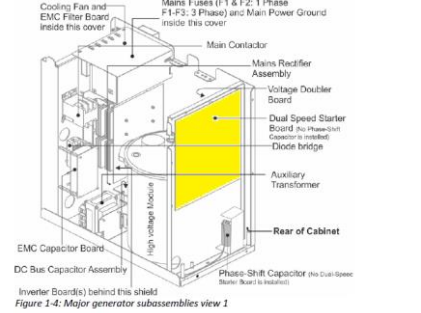
System Master		4.4CB800_01		
Power supply 230/24	0512-810-408	12-408		
Power supply 24V 8,3A	0073-810-033	12-033		
Isolator, Ethernet LAN 5000V	0073-810-036	08-200 4.4ISO01 4.5ISO01		
Power supply 20W	0073-810-126	12-026 4.4PS02 4.5PS02		

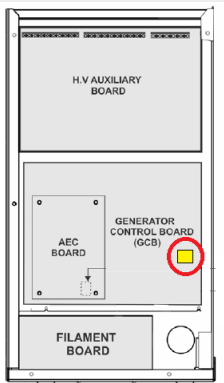
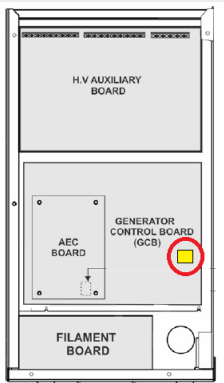
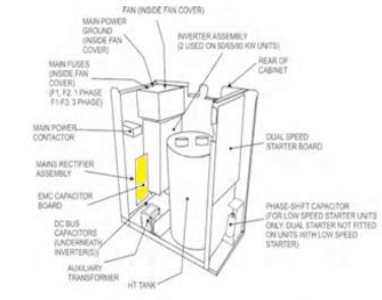
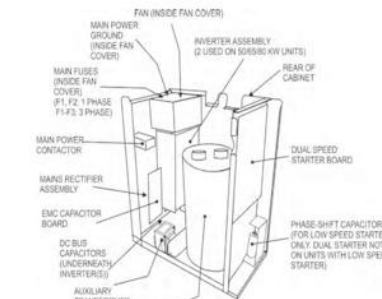
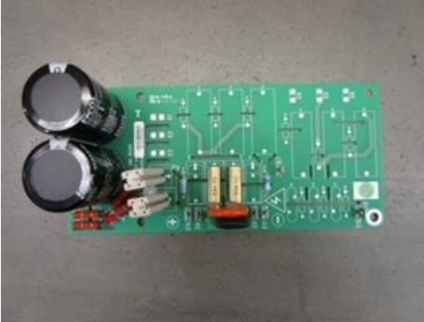
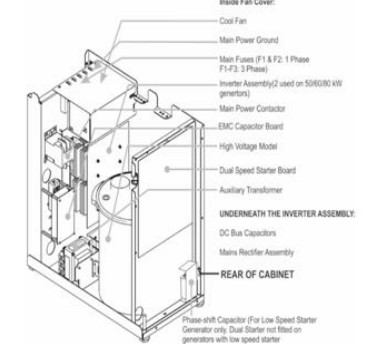
Splitter, Ethernet Y-adapter	0073-810-119	93-019 1.3SPL01 4.5SPL01		
Switch D-Link DGS-105/E	0073-810-020	93-020 4.5HUB02		
Filament Board	1100-810-001	71-955 SP90543100 CMP Replaces 73140700 and is backwards compatible.		
Inverter board For 80 kW generator	1100-810-002	90-479 SP90056910 <i>1pc also used in 50kW generators.</i>	 Figure 1-4: Major generator subassemblies view 1	
Inverter board For 65 kW generator	1100-810-003	90-481 SP73688511	 Figure 1-4: Major generator subassemblies view 1	

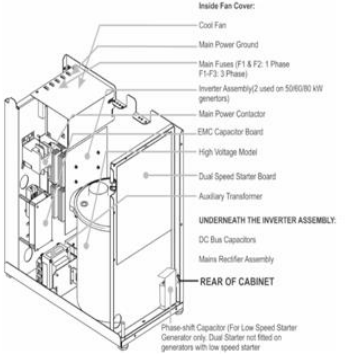
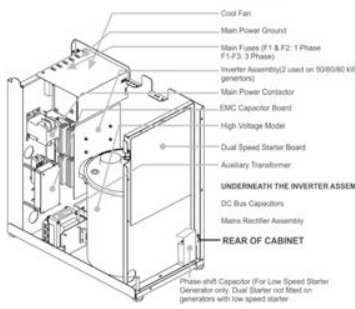
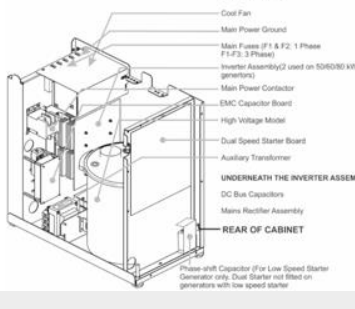
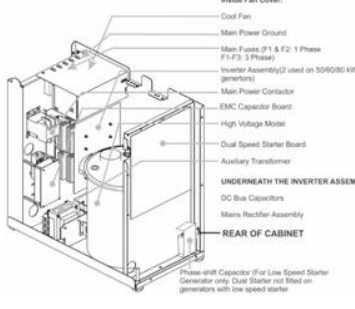
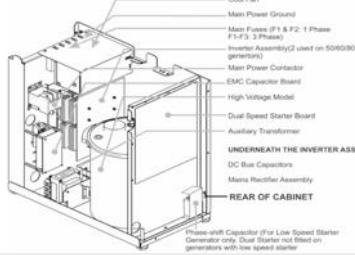
Inverter board For 50 kW generator	1100-810-004	SP73688512 2pcs used if installed in 50kW generator	 Figure 1-4: Major generator subassemblies view 1	
High voltage module assembly 50 kW	1100-810-008	90-432 90078620		
High voltage module assembly 65/80 kW	1100-810-009	90-437 SP90078607		
ION AEC board CMP200 Dedicated for AID	1100-810-013	71-956 73799800		

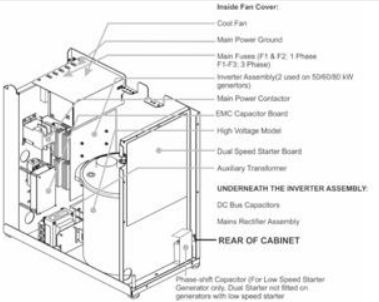
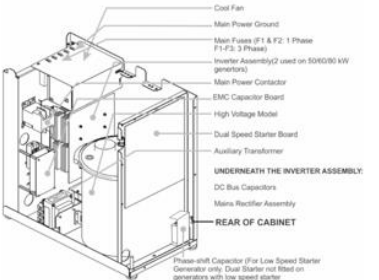
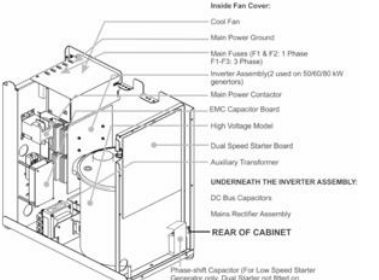
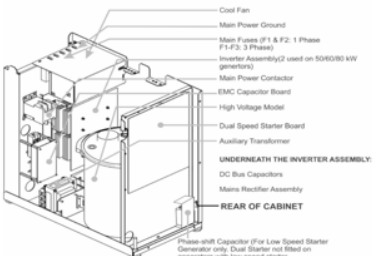
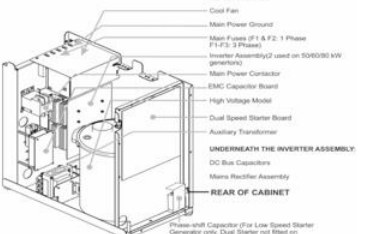
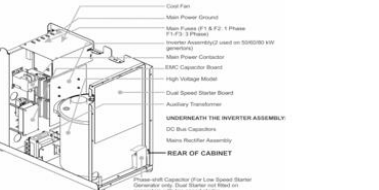
H.V. Auxiliary board CMP 200, 400/480 CMP200	1100-810-015	71-953 SP73944299 CMP		
Generator control board (GCB) CMP200	1100-810-208	71-983 90792600 Note! This board is backwards compatible with earlier versions of GCB's including 90114600.		
Dual speed starter subassembly CMP200 Kit no: 21	1100-810-017	90-017 SP90667221 CMP		
Dual speed starter subassembly CMP200 Kit no: 24				
Thermal sensor board	1100-810-022	SP90153100 CMP		

Contactor main power (DC) CMP200	1100-810-024	90-024 SP95068000 Replaced SP7220002 400CMP		
Contactor main power (AC) CMP200	1100-810-046	90-026 95065800 Note! Only compatible with Generator Control Board 90792600		
Exposure handle, external CMP200	1100-810-047	90-435 90007700		
Console cable, 15m / 50ft Console to Generator Interface PWB	1100-810-048	71-984 73209100 For CMP200 Generators		

Rectifier Assy mains	1100-810-025	71-985 6623071100 "Diode bridge"		
Transformer auxiliary	1100-810-026	903409-00	 Figure 1-4: Major generator subassemblies view 1	
Fan assembly	1100-810-027	71-959 739401-00	 Figure 1-4: Major generator subassemblies view 1	
Dual speed starter board	1100-810-028	71-989 728877-06 Has been replaced with SP90313202	 Figure 1-4: Major generator subassemblies view 1	
Dual speed starter board CMP200		SP90313202 Updated version with CAN-bus connection. Has been replaced with SP95129803		

Dual speed starter board CMP200		SP95129803 Current version, is compatible with and can be used in existing generators.		
Battery, Timekeeper SRAM	1100-810-032			
EMC capacitor board	n.a	73780309 Discontinued in 2014		
EMC capacitor board	1100-810-033	90187910 71-992 used in 3 phase 400/480V generators. 50/65/80kW (73780309 was used until 2014)		
Main line fuse for 50kW 400/480 V	1100-810-065	SP6713837500 71-982 F1, F2, F3 Qty: 3pc SC-60		

Main line fuses for 65/80 kW 400/480 V	1100-810-034	SP6800004300 71-987 F1, F2, F3 included (Kit qty: 3pcs) OTS60		
H.V. auxiliary board fuse	1100-810-035	5550037500 F1, F6, F7		
H.V. auxiliary board fuse	1100-810-036	5570210570 F2, F3		
H.V. auxiliary board fuse	1100-810-037	5570211500 F10		
H.V. auxiliary board fuse	1100-810-038	5550033400 F11		

H:V: auxiliary board fuse Generator control board fuse	1100-810-039	550032600 Aux: F12 GCB: F2		
H.V. auxiliary board, 400/480V unit fuse	1100-810-041	6711905800 71-986 (1pc) F4, F5		
H.V. auxiliary 400/480V unit fuse	1100-810-042	5550005300 71-xxx (1pc) F8, F9		
Filament board fuse	1100-810-043	5550033100 F1 F2		
DSS board fuses	1100-810-044	6739951800 F1 F2		
Generator control board fuse	1100-810-045	5550033600 F3		

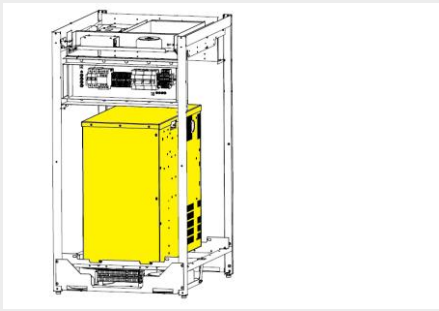
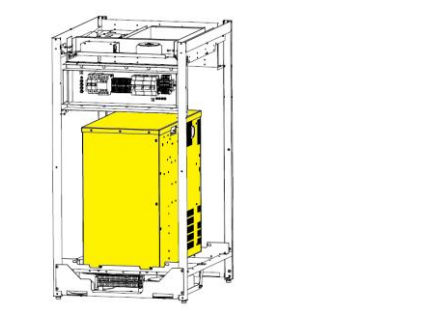
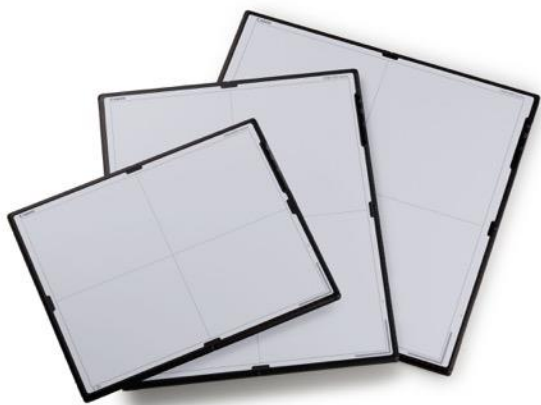
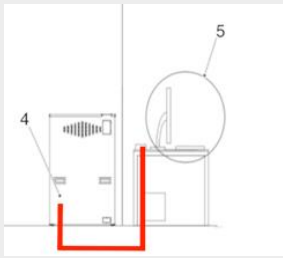
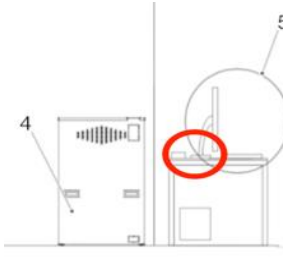
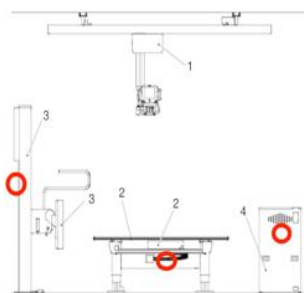
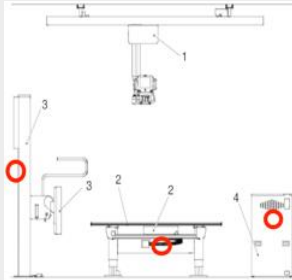
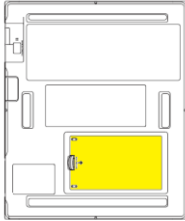
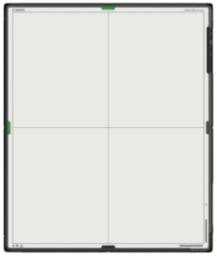
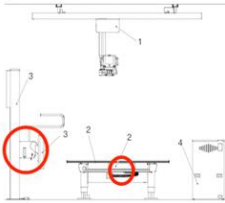
Generator CMP200 50kW	1100-810-100	90-953 Incl. DAP- license		
Generator CMP200 65kW	1100-810-101	90-955 Incl. DAP- license		
Generator CMP200 80kW	1100-810-102	90-957 Incl. DAP- license		

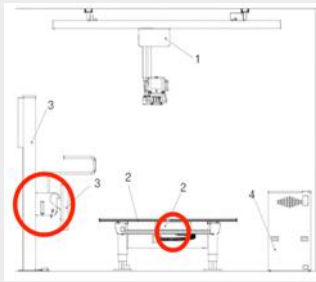
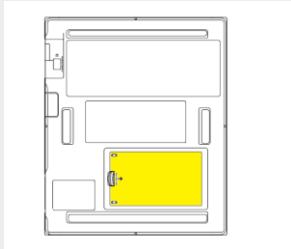
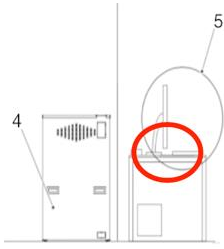
Image Acquisition System & Parts in Control room

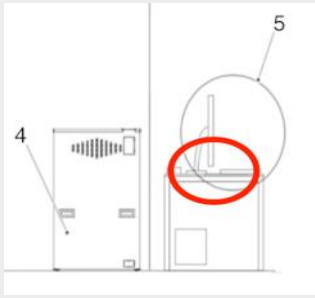
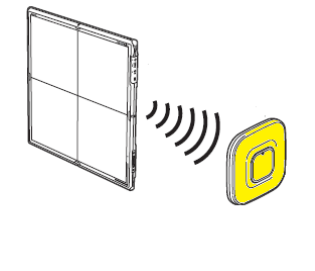
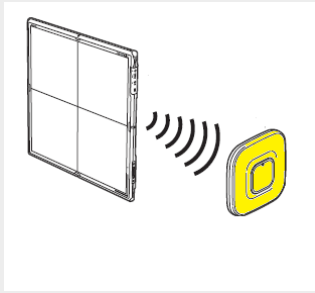
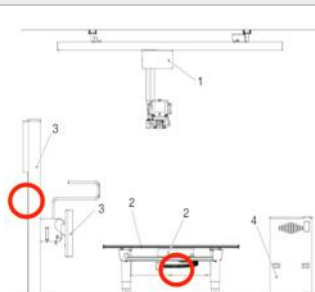
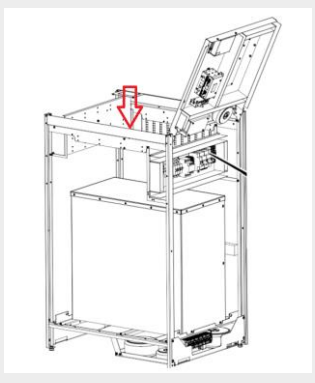
Note! Some parts related to the Image Acquisition System is not available for sales in USA and Canada. Please refer to the Spare part list for availability.

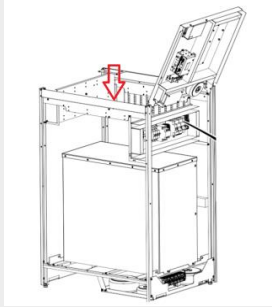
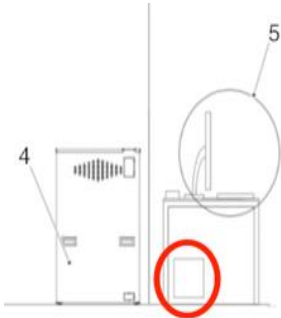
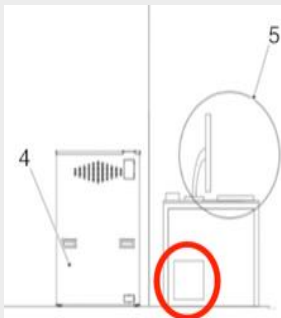
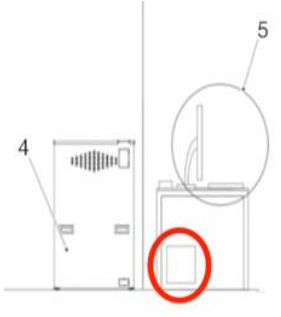


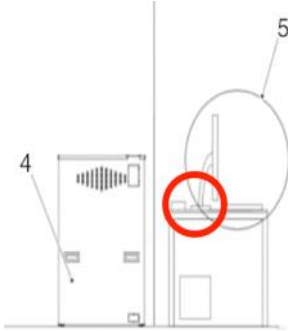
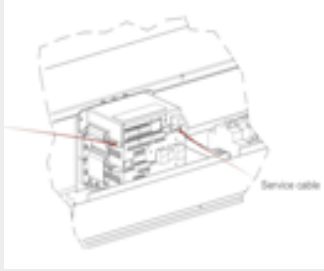
Description	Spare Part article no	Info/Number on spare part	Drawing	Photo
Cable exposure handle, 15 meter	0073-810-026	0073-750-026 <i>Cable routed from System Cabinet to Mini console</i>		
Mini console	1105-810-436	90-436		
CXDI-710C Wireless (incl. 2 battery packs)	1100-810-051	90-702		
CXDI-810C Wireless (incl. 2 battery packs)	1100-810-052	90-578		
CXDI-410C Wireless (incl. 2 battery packs)	1100-810-053	90-677		
Multi box MB-4A	1100-810-055	90-694 <i>Located either</i> 1) <i>on the back of Wallstand</i> 2) <i>Underneath Table bucky</i> 3) <i>Inside system cabinet</i>		

Multi box MB-02	1100-810-761	90-761		
Cover unit, Battery	1100-810-070	90-726 BM7-2329-000		
Corner Cushions 4pcs	1100-810-071	90-725 BA5-5135-000 (adhesive) For x10 series detectors		
Corner Cushions 4pcs	1100-810-072	90-728 BA5-8061-000 (interlocking) For 710C and 810C Wireless only		
Cover Sheet UIF, adhesive	1100-800-073	90-727 BA5-5284-000 For x10 series detectors		
Wiring cable WC-4A	1100-810-056	90-587 <i>For x10 series detectors and MB-4A</i>		

Wiring cable WC-01-07 (7,5m)	1100-810-057	90-763 <i>For Elite/Pro-detectors and MB-02</i>		
Status indicator SI-4A	1100-810-057	90-695		
Battery pack LB-4A	1100-810-058	90-586		
Battery Charger BC-1A 230V	1100-810-059	90-662		
Battery Charger BC-01 230V		1000-099-202 90-760		
Ready Indicator RI-3A	1100-810-060	90-688		

IR check-in I/F link module	1100-810-061	90-583		
Access point Aruba 505	1100-810-062	90-910 Not available, has been replaced by 1100-810-063		
Access point Netgear WAX610 AP	1100-810-063	90-911 Has replaced p/n 1100-810-062.		
CXDI Power box PB-6	1100-810-063	90-663		
Switch, POE D-Link DGS-1008P	1100-810-285	93-002		

Switch, POE D-Link DGS-1008P/F	1100-810-118	93-018		
HP Z4 G5 WS 3DISK T400 Standard Workstation	1100-810-064	90-736		
HP Z4 G5 WS 3DISK RTX4000 INR Workstation	1100-810-067	90-737		
Network interface GB-Ethernet For 515NE302 (90-746) ETH 3	1100-810-749	90-749		
Eizo RadiForce MS236WT-BK 23" Multitouch Monitor EU	1100-810-087	90-507		

Emergency stop, external	--	0175-154-100		
Service cable, 5 meters	1100-810-201	0065-750-410		

END OF DOCUMENT