

# SYSTEM SOFTWARE VERSIONS

# SYSTEM SOFTWARE


> HOW CAN I CHECK THE SOFTWARE VERSION IN THE SYSTEM?

| SOFTWARE  |         |                         |               |         |          |                     |
|-----------|---------|-------------------------|---------------|---------|----------|---------------------|
| Canon NE  | NERF    | CXDI NE-ETP<br>Overwrap | CAN<br>Device | Genware | Firmware | Arcoma System<br>SW |
| 3.11.2.12 | 3.1.3.2 | 4.1.1.0                 | 1.5           | AK/1.15 | AT       | 1.2.0               |
| CANON     |         | GENERATOR (CPI)         |               |         | ARCOMA   |                     |



# SYSTEM SOFTWARE

| SOFTWARE  |         |                         |               |         |          |                     |
|-----------|---------|-------------------------|---------------|---------|----------|---------------------|
| Canon NE  | NERF    | CXDI NE-ETP<br>Overwrap | CAN<br>Device | Genware | Firmware | Arcoma System<br>SW |
| 3.11.2.12 | 3.1.3.2 | 4.1.1.0                 | 1.5           | AK/1.15 | AT       | 1.2.0               |
| CANON     |         | GENERATOR (CPI)         |               |         | ARCOMA   |                     |

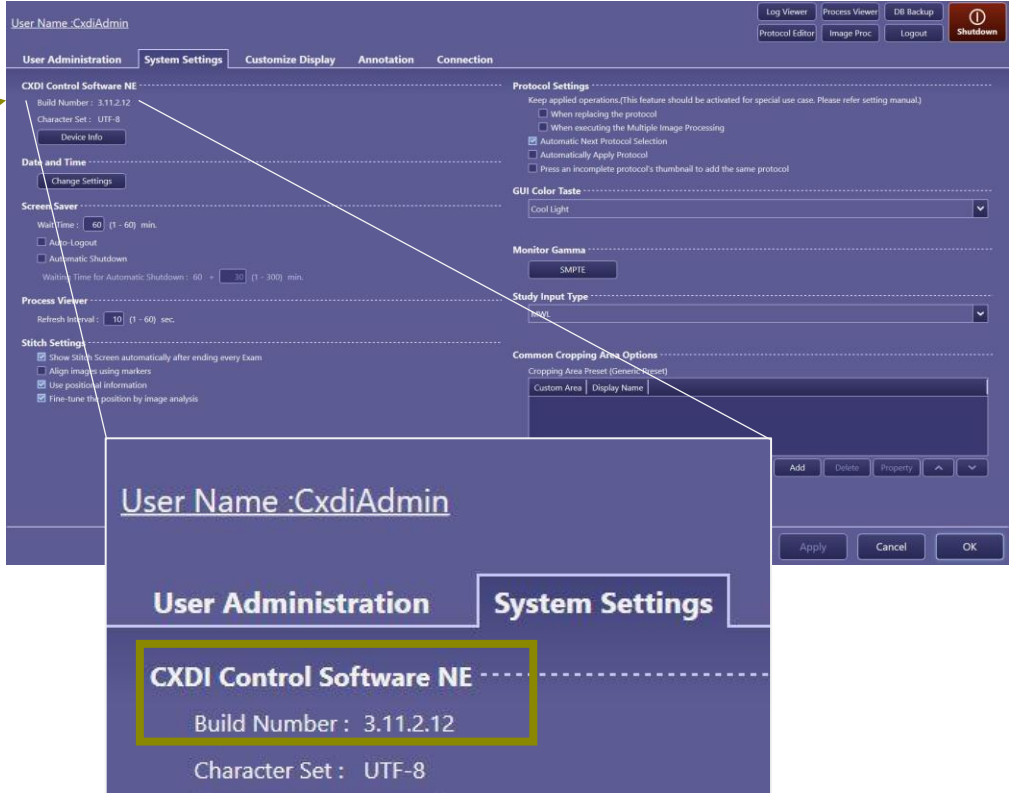

# SYSTEM SOFTWARE

| SOFTWARE  |         |                         |               |         |          |                     |
|-----------|---------|-------------------------|---------------|---------|----------|---------------------|
| Canon NE  | NERF    | CXDI NE-ETP<br>Overwrap | CAN<br>Device | Genware | Firmware | Arcoma System<br>SW |
| 3.11.2.12 | 3.1.3.2 | 4.1.1.0                 | 1.5           | AK/1.15 | AT       | 1.2.0               |
| CANON     |         | GENERATOR (CPI)         |               |         | ARCOMA   |                     |

This PC > Windows (C:) > CCS-S

Name

- Canon.Medical.DR.JointController.ServerProxy.dll
- Canon.Medical.DR.JointController.SharedData.dll
- Canon.Medical.DR.JointController.SharedLibrary.dll
- Canon.Medical.DR.ServiceTool.APSetting.dll
- Canon.Medical.DR.ServiceTool.UI.CollectionTool.dll
- Canon.Medical.DR.ServiceTool.UI.Detector.dll
- Canon.Medical.DR.ServiceTool.UI.DMLisReg.dll
- Canon.Medical.DR.ServiceTool.UI.DmLisReg.Utility.dll
- Canon.Medical.DR.ServiceTool.UI.FerryTypeFPDMaint.dll
- Canon.Medical.DR.ServiceTool.UI.FerryTypeFPDMaint.dll
- Canon.Medical.DR.ServiceTool.UI.GammaAdjustment.dll
- Canon.Medical.DR.ServiceTool.UI.GammaAdjustment.dll
- Canon.Medical.DR.ServiceTool.UI.GammaAdjustment.dll
- Canon.Medical.DR.ServiceTool.UI.InputChecker.dll
- Canon.Medical.DR.ServiceTool.UI.MigrationTool.dll
- Canon.Medical.DR.ServiceTool.Utility.FileConverter.dll
- Canon.Medical.DR.Seychelle.Component.AuditLog.dll
- Canon.Medical.DR.Seychelle.Component.CruiseGCO.dll
- Canon.Medical.DR.Seychelle.Component.DrawObject.dll
- Canon.Medical.DR.Seychelle.Component.FilmComp.dll
- Canon.Medical.DR.Seychelle.Component.Filming.dll
- Canon.Medical.DR.Seychelle.Component.FilmIP.dll

Canon.Medical.DR.JointController.ServerProxy.dll Prop...

General Security Details Previous Versions

| Property          | Value                                     |
|-------------------|-------------------------------------------|
| Description       |                                           |
| File description  | Canon.Medical.DR.JointController.Serve... |
| Type              | Application extension                     |
| File version      | 3.1.3.2                                   |
| Product name      | Canon.Medical.DR.JointController.Serve... |
| Product version   | 3.1.3.2                                   |
| Copyright         | Copyright CANON INC. 2024                 |
| Size              | 422 KB                                    |
| Date modified     | 7/9/2024 2:23 PM                          |
| Language          | Language Neutral                          |
| Original filename | Canon.Medical.DR.JointController.Serve... |

[Remove Properties and Personal Information](#)

OK Cancel Apply

# BiAA – SYSTEM SOFTWARE

| SOFTWARE  |         |                      |            |         |          |                  |
|-----------|---------|----------------------|------------|---------|----------|------------------|
| Canon NE  | NERF    | CXDI NE-ETP Overwrap | CAN Device | Genware | Firmware | Arcoma System SW |
| 3.11.2.12 | 3.1.3.2 | 4.1.1.0              | 1.5        | AK/1.15 | AT       | 1.2.0            |
| CANON     |         | GENERATOR (CPI)      |            |         | ARCOMA   |                  |

CXDI\_NE\_Overwrap

Name

ExposureCondition.exe.config

ExposureCondition.exe

GenConfig.exe

## CPI GenConfig

Generator Control Settings

General

Receptors

Network

Licensed Features

Version Information

4.1.1.0

CPI Generator Config [CLINICAL RELEASE]

Generator Configuration

Generator Type

CMP200

▼

COM Port

COM1

▼

DAP Display Units

mGycm²

▼

Logging Settings

Level

Verbose

▼

Export Logs

Generator Control Software is not running


Exit

# BiAA – SYSTEM SOFTWARE

| SOFTWARE  |         |                         |               |         |          |                     |
|-----------|---------|-------------------------|---------------|---------|----------|---------------------|
| Canon NE  | NERF    | CXDI NE-ETP<br>Overwrap | CAN<br>Device | Genware | Firmware | Arcoma System<br>SW |
| 3.11.2.12 | 3.1.3.2 | 4.1.1.0                 | 1.5           | AK/1.15 | AT       | 1.2.0               |
| CANON     |         | GENERATOR (CPI)         |               |         | ARCOMA   |                     |



PRESS THE WHEEL TO ACCESS  
THE SERVICE/SETTINGS TAB.  
PASS KEY: 1895

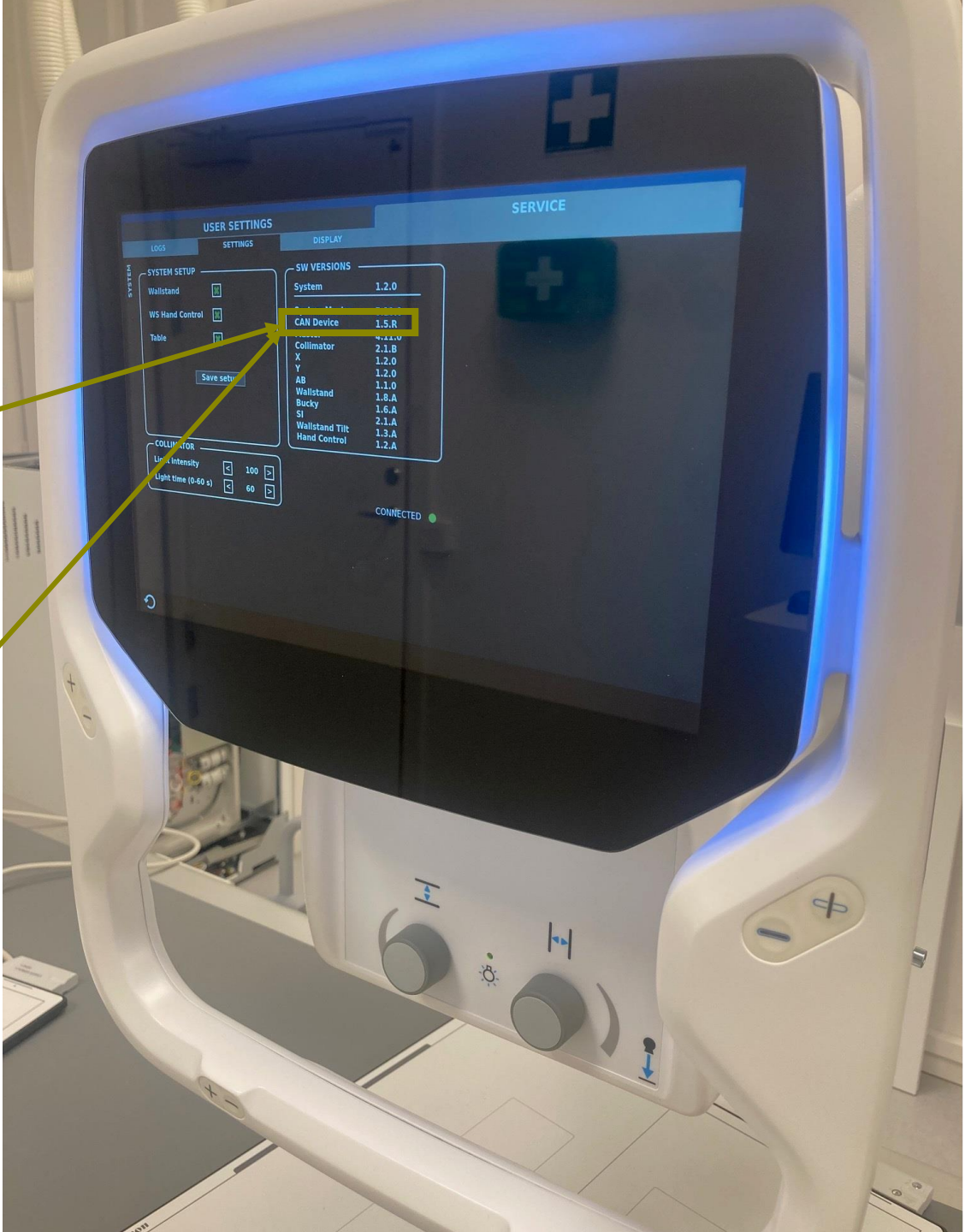

# BiAA – SYSTEM SOFTWARE



| SOFTWARE  |         |                      |            |         |          |                  |
|-----------|---------|----------------------|------------|---------|----------|------------------|
| Canon NE  | NERF    | CXDI NE-ETP Overwrap | CAN Device | Genware | Firmware | Arcoma System SW |
| 3.11.2.12 | 3.1.3.2 | 4.1.1.0              | 1.5        | AK/1.15 | AT       | 1.2.0            |
| CANON     |         | GENERATOR (CPI)      |            |         | ARCOMA   |                  |

About

GenwareMP Generator Utility

Revision Number: 1.15

Power Unit - CMP200DR

Revision Number: 2.55

Part Number: 90135000AT

Build Number: 016B64361B

Engineering Number: 0

SC Revision Number: 111003

Features

- 1 DAP.
- 2 Interface Option 27.
- 3 Flat Panel AEC

This Product is subject to license agreements.

Close

# BiAA – SYSTEM SOFTWARE



| SOFTWARE  |         |                      |            |         |          |                  |
|-----------|---------|----------------------|------------|---------|----------|------------------|
| Canon NE  | NERF    | CXDI NE-ETP Overwrap | CAN Device | Genware | Firmware | Arcoma System SW |
| 3.11.2.12 | 3.1.3.2 | 4.1.1.0              | 1.5        | AK/1.15 | AT       | 1.2.0            |
| CANON     |         | GENERATOR (CPI)      |            |         | ARCOMA   |                  |

About

GenwareMP Generator Utility

Revision Number: 1.15

Power Unit - CMP200DR

Revision Number: 2.55

Part Number: 90135000AT

Build Number: 016B64361B

Engineering Number: 0

SC Revision Number: 111003

Features

1 DAP.

2 Interface Option 27.

3 Flat Panel AEC


This Product is subject to license agreements.

Close

# BiAA – SYSTEM SOFTWARE

| SOFTWARE  |         |                         |               |         |          |                     |
|-----------|---------|-------------------------|---------------|---------|----------|---------------------|
| Canon NE  | NERF    | CXDI NE-ETP<br>Overwrap | CAN<br>Device | Genware | Firmware | Arcoma System<br>SW |
| 3.11.2.12 | 3.1.3.2 | 4.1.1.0                 | 1.5           | AK/1.15 | AT       | 1.2.0               |
| CANON     |         | GENERATOR (CPI)         |               |         | ARCOMA   |                     |

CHECK SW VERSIONS IN RELEASE  
NOTE/DOCUMENTATION.

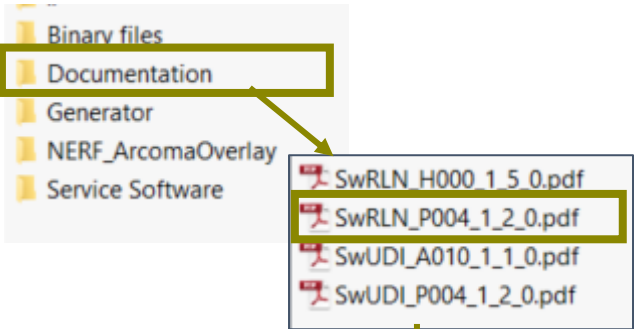


3.1 1.2  
3.1.1 Software versions

| Product      | Part            | Software Version |
|--------------|-----------------|------------------|
| A001         | SystemMaster    | 5.10.0           |
|              | Master          | 4.11.0           |
|              | UI/Collimator   | 2.1.B            |
|              | Guard           | 0.10.R           |
|              | SI              | 2.1.A            |
|              | X               | 1.2.0            |
|              | Y               | 1.2.0            |
|              | Alpha/Beta (AB) | 1.1.0            |
|              | Bucky           | 1.6.A            |
|              | Wallstand       | 1.8.A            |
| A002         | Wallstand tilt  | 1.3.A            |
|              | HandControl     | 1.2.A            |
|              | SSW             | 1.12.1           |
| A008         | OTC-Display     | 5.2.0            |
| A010         | Arcoma Overlay  | 1.2.1            |
| Canon NE     |                 | 3.11.2.12        |
| Canon NERF   |                 | 3.1.3.2          |
| CPI Overwrap |                 | 4.1.1.0          |

# BiAA – SYSTEM SOFTWARE

| SOFTWARE  |         |                         |               |         |          |                     |
|-----------|---------|-------------------------|---------------|---------|----------|---------------------|
| Canon NE  | NERF    | CXDI NE-ETP<br>Overwrap | CAN<br>Device | Genware | Firmware | Arcoma System<br>SW |
| 3.11.2.12 | 3.1.3.2 | 4.1.1.0                 | 1.5           | AK/1.15 | AT       | 1.2.0               |
| CANON     |         | GENERATOR (CPI)         |               |         | ARCOMA   |                     |

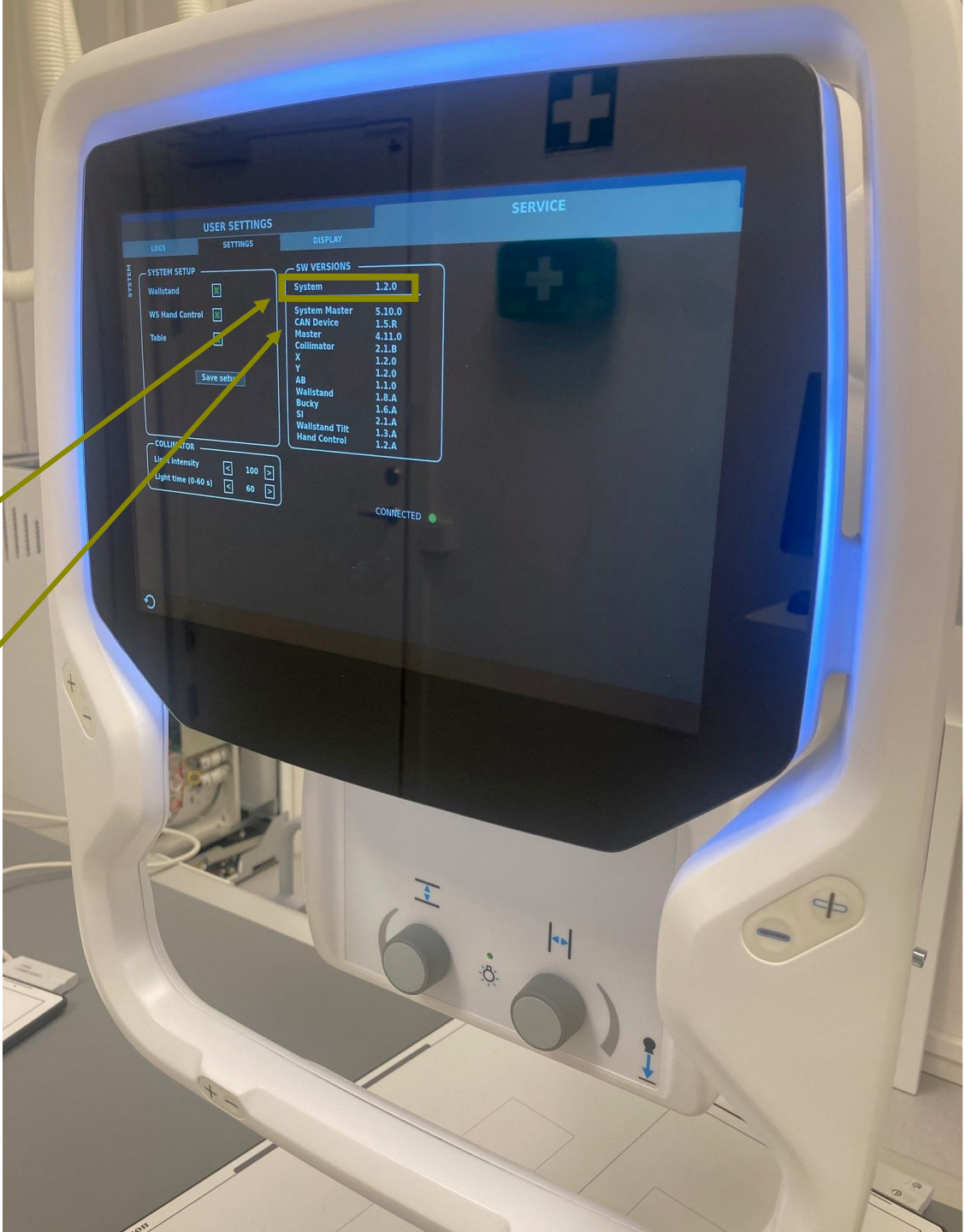

PRESS THE WHEEL TO ACCESS  
THE SERVICE/SETTINGS TAB.

PASS KEY: 1895

# THANK YOU

Arcoma AB | Annavägen 1 | 352 46 Växjö | Sweden



**PHONE**

+46 470 70 69 00



**EMAIL**

office@arcoma.se



**WEBSITE**

[www.arcoma.se](http://www.arcoma.se)