

4. CLEANING OF HANDLE FRAME AND DISPLAY

General recommendations of disinfectants

For cleaning or disinfections of the handle frame and the display, it is recommended to use an **alcohol only disinfectant**.



Cleaning agents containing for instance the below listed compounds should be avoided:

- quaternary ammonium compounds (e.g. benzalkonium chloride, alkyl dimethyl benzyl ammonium chlorides and alkyl dimethyl ethylbenzyl ammonium chlorides)
- L-lactic acid
- Citric acid
- pH adjusting compounds and stabilizers (commonly present in disinfectants containing hydrogen peroxide)

Examples of compatible disinfectants

Ethanol based wipes:

- Wet Wipe Ethanol Disinfection With Detergent
<https://wetwipe.dk/wet-wipe-ethanol-disinfection-med-saebe/?lang=da>
- Wet Wipe Ethanol Disinfection without detergent
<https://wetwipe.dk/wet-wipe-ethanol-disinfection-uden-saebe/?lang=da>
- Ultra Swipes (Canada)
<https://germiphene.com/product/ultra-swipes/>

Ethanol based liquid disinfectant

- Liv DES +72
Ytdesinfektion rengöring

Application of disinfectant

It is recommended to use only a moderate amount of cleaning liquid. Either use disinfectant wipes or moisten a soft, lint-free cloth or tissue with a moderate amount disinfectant and gently wipe the surface clean. Liquid must not be sprayed directly over the surface to be disinfected.