


# STITCHING FRAME

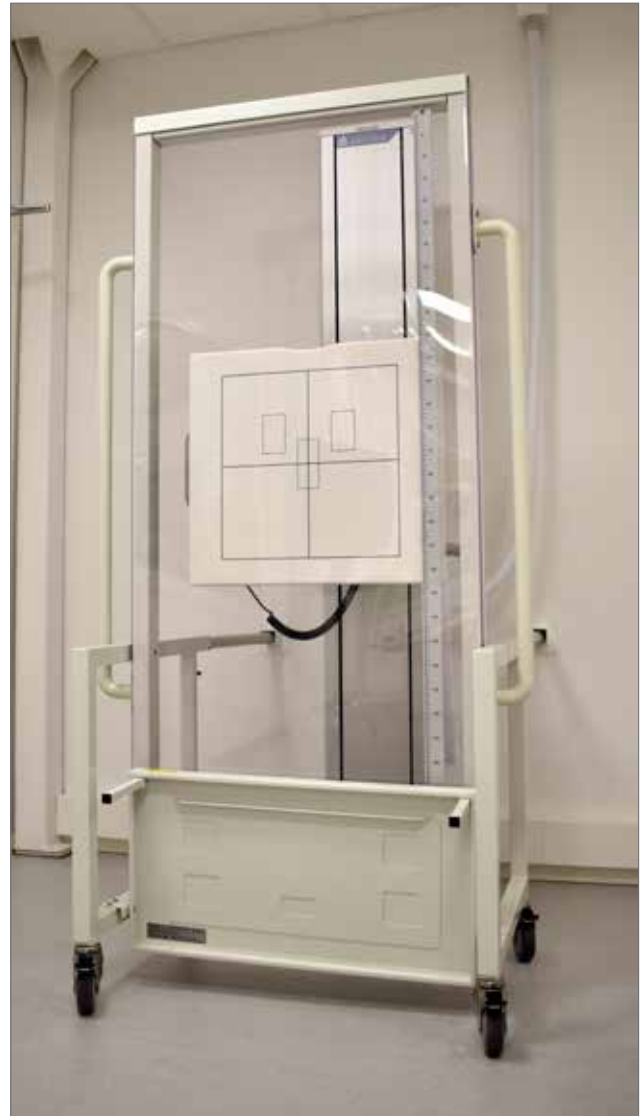

The stitching frame can be used during stitching with the Arcoma Intuition and Arcoma Precision i5 systems and will assist the patient retaining a stable position during x-ray.

The stitching frame is easy to move and position, and the link-arms provide a flexible placement of the screen in front of the wall stand. The footstep is integrated with the screen and can easily be moved and folded up when needed.

# SPECIFICATIONS

## DIMENSIONS

Height: 1985 mm

Width: 1074 mm

Depth: 1010 mm – 760 mm (250mm adjustable)

Weight: 40 kg

Foot stool height: 190 mm

## CLEANING AND DISINFECTION

Wipe the Stitch Frame with a piece of soft cloth soaked in water containing neutral detergent and wrung dry. DO NOT use abrasive or chemical agents or detergents containing solvents.

For infection prevention is advised to disinfect the Perspex plate of the Stitch Frame. Soak a piece of soft cloth with ethanol 80~90 %, or glutaraldehyde solution and wipe the Perspex surface thoroughly when a stitch procedure for a patient has been finished.

*Specifications subject to change without notice*